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# Content

- 2 Explanation of Symbols
- 4 Products
- 18 The drainova® Reservoir soft-vacuum
- 18 Warnings
  - in relation to your catheter
- 20 Connecting the drainova® Reservoir
- 23 Final steps
- 23 Emptying the drainova® Reservoir
- 24 Returns/refunds
- 24 Glossary

en

**Please carefully read the instructions before using the drainova® Reservoir soft-vacuum.**

## You have questions or need help?

Contact us:

DE: +49 7471 73972-0  
info@ewimed.com

AT: +43 2231 22500  
info@ewimed.com

EN: +49 7471 73972-0  
info@ewimed.com

SE: +46 8 25 11 69  
info@fenik.se

CH: +41 52 577 02 55  
info@fenik.ch

HU: +36 1 998 8295  
info@ewimed.hu

**For further detailed information please visit our website  
[www.ewimed.com](http://www.ewimed.com)**

# The drainova® Reservoir soft-vacuum

The drainova® Reservoir soft-vacuum contains sterile material required for the safe and easy drainage of effusion fluids with your catheter.

The drainova® Reservoir soft-vacuum will hereinafter be referred to as "drainova® Reservoir". The drainova® Reservoir is CE labelled and have been sterilized with ethylene oxide (see individual packaging).

en

## Warnings

### general

#### **Do not use this product if:**

- the sterile packaging is damaged
- damaged
- the expiry date has elapsed

#### **The drainage consumables are intended for single use only!**

The drainova® Reservoirs are delivered in sterile packaging which have undergone quality controls. The drainova® Reservoir can not be re-sterilised.

The product is intended for single use only. Re-use could contribute to a cross-contamination.

ewimed GmbH accepts no liability for re-used, re-sterilised products or products taken from damaged packaging.

#### **Drainage of the chest cavity**

Only drain the amount of fluid prescribed by your doctor.

ewimed does not recommend draining more than 1000 ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in order to pause the flow.

Possible complications relating to repeated drainage of pleural effusions\* include pneumothorax\*, re-expansion pulmonary oedema\*, hypotension\*, circulatory collapse\*, septa\* in the chest cavity or infection\*.

#### **Drainage of the peritoneal cavity**

Only drain the amount of fluid prescribed by your doctor.

ewimed does not recommend draining more than 2000 ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in order to pause the flow.

Possible complications relating to repeated drainage of ascites\* include hypotension\*, circulatory collapse\*, electrolyte imbalance\*, loss of protein\*, hepatorenal syndrome\*, leakage of ascites from incision sites, peritonitis\*, septa\* in the peritoneal cavity\* and infection.


### Position of the drainova® Reservoir during drainage

The drainage of fluid is assisted by the force of gravity. Therefore please ensure that the drainova® Reservoir is always positioned under the catheter exit site whilst draining, e.g. on the floor.

### Cleaning / disinfection

Should fluid leak out and come into contact with skin, clean the skin with soap and water. Use suitable cleaning or disinfectant products for surfaces. Do not use Octenisept® or products containing iodine to clean/disinfect the catheter or safety valve.

Please observe the reservoir change intervals according to the hygiene regulations of your institution.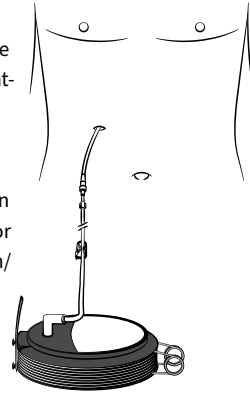

en

## Warnings


in relation to your catheter

### Never cut through the catheter or cut the safety valve off.

Ensure that the catheter does not come into contact with any sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i.e. is no longer attached), proceed as follows:

- firmly pinch the catheter closed with your fingers
- using your other hand, hold the emergency slide clamp open so that the catheter can be fed through the open end of the clamp.
- push the catheter completely through the narrow end of the clamp.
- Immediately inform a doctor.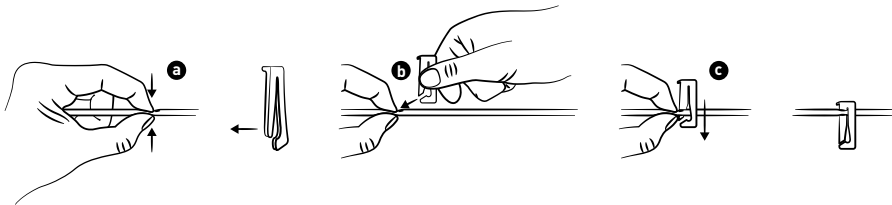

### Only use original accessories.

Do not attempt to connect the catheter to non-original drainage products. Only the specially-developed access tip can be inserted into the safety valve. The use of foreign items can damage the safety valve. Incorrect handling can cause air to enter the body or effusion fluid to leak out.

### Always keep the catheter safety valve clean and the access tip on the connected tube sterile.

Take care to ensure that they do not come into contact with any un-sterile objects in order to avoid contamination. The safety valve must be disinfected before each drainage.

### While draining, make sure that the safety valve and the access tip are securely connected. Take care not to tug or pull on the connected tube.

"Take particular care to ensure that the access tip is inserted in the safety valve in a sterile manner and make sure that the connection is secure. It can lead to contamination if the parts become disconnected by mistake. In case this occurs, disinfect the safety valve with a new alcohol wipe and dispose of the disconnected drainage material. Use a new drainova® Reservoir to continue with the drainage. The drainova® clickFix can be used to prevent unintentional disconnection."

The drainova® clickFix is not contained in the drainova® Reservoir but can be ordered separately.

## Connecting the drainova® Reservoir

1. Remove the drainova® Silicone Cap or the drainova® Protective Cap:


- **Using the drainova® Silicone Cap:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® Silicone Cap. Dispose of the cap.


- **Using the drainova® Protective Cap:**

Firmly hold the safety valve of the catheter, rotate the drainova® Protective Cap anticlockwise and carefully remove. Dispose of the cap.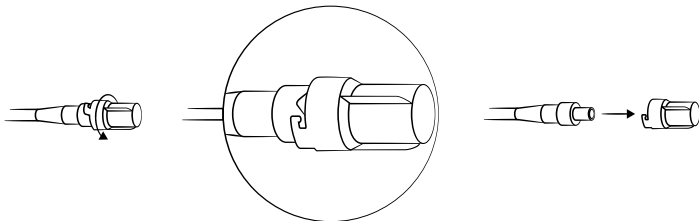


2. Hold the safety valve in one hand and clean the area around the safety valve-opening with an alcohol wipe or a suitable alcohol-based disinfectant.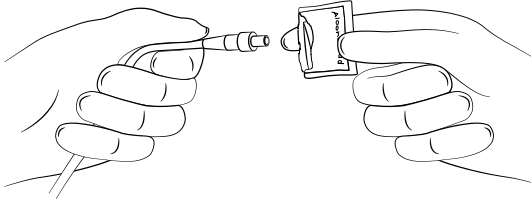


**Warning:** Do not use Octenisept® or other disinfectants containing iodine.

en

3. Open the packaging of the drainova® Reservoir and take it out.  
The product in the packaging is sterile. It is important that the access tip on the tubing remains sterile.
4. Remove the tape wrapped around the coiled up tubing.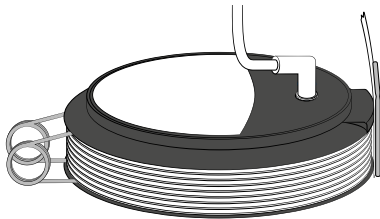


5. Hold the base of the access tip in one hand. Completely close the pinch clamp on the tube with the other hand.

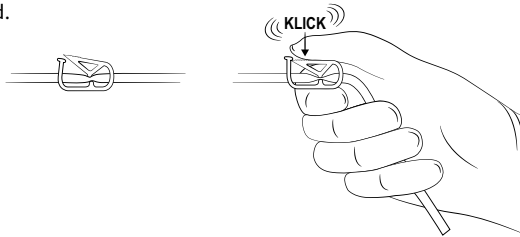
6. Continue to hold the access tip at the base and carefully remove the protective film from the access tip (if necessary by gently rotating).

- 6.1 Insert the access tip in the safety valve. Ensure that the safety valve and the access tip are securely connected during drainage. A click can be heard and felt when connected.

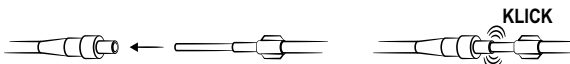
7. Push down on the drainova® Reservoir to remove the orange-coloured safety clasp. Keep the orange-coloured clasp for later disposal of the reservoir.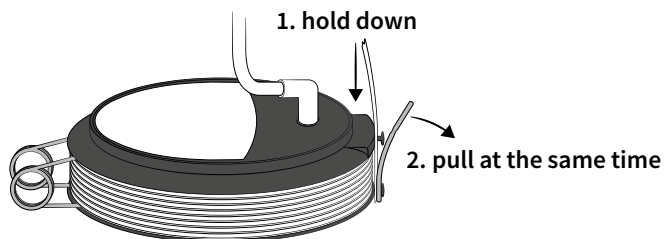

**Recommendation:** The duration of drainage can be shortened by placing the drainova® Reservoir at a lower height e.g. on the floor.


8. Open the pinch clamp on the tubing so that drainage can begin.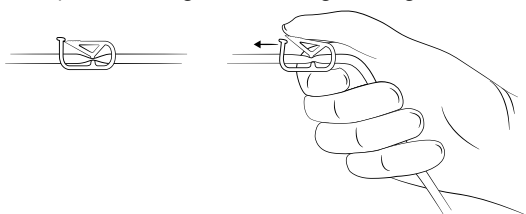

**Please note:** The flow can be slowed or interrupted when:


- The fluid has almost completely drained
- The fluid has a high viscosity
- The safety valve is blocked due to a clot\* or a protein thread

9. When you have reached the drainage amount recommended by your doctor, stop drainage by closing the pinch clamp. The top stop button shows you the drained amount.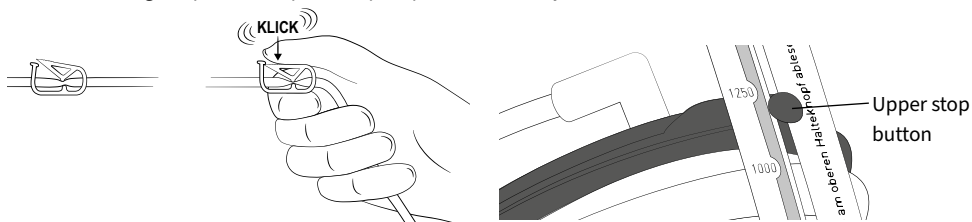

## Final steps

1. Open the packaging of the drainova® Silicone Cap or the PleurX™ Protective Cap. But to begin with, leave the cap in the packaging.
2. Take the tubing in one hand and the safety valve in the other hand. Pull the access tip in a firm, smooth motion out of the safety valve. Place the tubing down.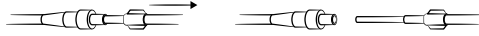


3. Clean the safety valve for approx. 30 seconds with an alcohol wipe or other suitable alcohol-based disinfectant and then dispose of the used wipe.

**Warning:** Do not use Octenisept® or other disinfectants containing iodine.


en

4. Attach either the drainova® Silicone Cap or PleurX™ Protective Cap:
  - **Using the drainova® Silicone Cap:**  
Securely attach the new sterile Drainova® Silicone Cap onto the safety valve of the catheter. The cap will stay in place without the need for additional support.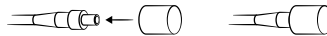

- **Using the PleurX™ Protective Cap:**

Attach the new PleurX™ Protective Cap onto the safety valve and turn clockwise until it clicks into place.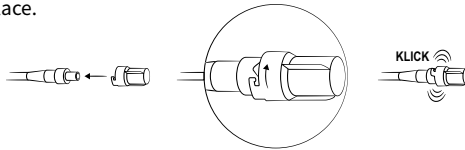

## Emptying the drainova® Reservoir

1. For the following steps you will need scissors, a towel, the orange-coloured safety clasp, a plastic bag (e.g. the outer packaging in which the drainova® Reservoir came in).
2. Place the drainova® Reservoir on the floor. Open the pinch clamp and lift up the tube. Residual fluid in the tube can now drain off.
3. Close the pinch clamp.
4. Cut through the tube underneath the one-way valve\* (illustration 1). Dispose of the cut-off tube piece with the pinch clamp and the access tip. If necessary use the towel laid out ready. In order to flush the fluid in the reservoir down the toilet, position the tubing with the opening facing downwards over the toilet.

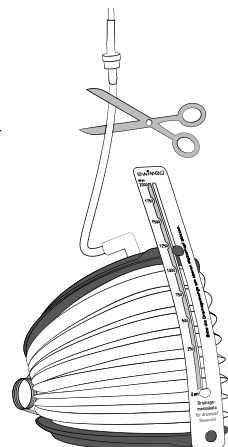
illustration 1

**Warning:** Used products and effusion fluid may pose biological risks. Therefore please observe the local and national regulations when disposing of used products and fluid.

5. Start the emptying process by compressing the reservoir. When the reservoir is nearly empty it may also be helpful to hold it upside down.
6. As soon as the fluid has been emptied and the reservoir is back in the compressed position, you can apply the orange-coloured safety clasp. Pack the reservoir with the piece of tube in the plastic bag laid out ready.
7. Dispose of the bag in accordance with the local regulations.

en

## Returns / refunds

Only originally packaged (unopened, non-damaged, non-labelled) boxes of drainova® Reservoir will be accepted for return and refund.

## Glossary

<b>Ascites</b>	Accumulation of fluid in the abdomen
<b>Circulatory collapse</b>	Temporary loss of consciousness
<b>Clot</b>	Blood clot
<b>DEHP</b>	Softening agents in plastics
<b>Electrolyte imbalance</b>	Imbalance of ions (e.g. natrium, potassium, calcium) in the blood or somatic cells
<b>Erythema</b>	Skin redness
<b>Hepatorenal syndrome</b>	Impaired renal function
<b>Hypotension</b>	Low blood pressure
<b>Incision</b>	Skin incision during the operation e.g. for the implantation of the catheter
<b>Infection</b>	Reaction of the body's own defence mechanism to germs e.g. bacteria, viruses etc.
<b>Loss of protein</b>	Reduction in the amount of protein in the blood (protein depletion)

<b>Oedema</b>	Swelling due to excessive accumulation of fluid
<b>One-way valve</b>	Non-return system for increased patient safety
<b>Peritoneal cavity</b>	Part of the abdominal cavity
<b>Peritonitis</b>	Infection of the peritoneum
<b>Phthalate</b>	Softening agents (e.g. DEHP)
<b>Pleural effusion</b>	Effusion (accumulation of fluid) in the pleural cavity
<b>Pneumothorax</b>	Air in the area between the lung and the chest wall
<b>Re-expansion - Pulmonary oedema</b>	Fluid retention in functional lung tissue due to sudden expansion of the previously compressed lung; acute shortness of breath
<b>Septa</b>	Tissue bridging / building of chambers between separated areas in a body cavity (e.g. abdomen)
<b>Subcutaneous tissue</b>	Tissue beneath the skin

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PleurX™ is a trademark and /or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and /or registered trademark of ewimed.
Shaoxing Yibon Medical Co., Ltd.
OBELIS S.A.
**Sweden** | fenik AB | Ekbacksvägen 28 | 168 69 Bromma | Tel.: +46 825 1169 | [info@fenik.se](mailto:info@fenik.se)