
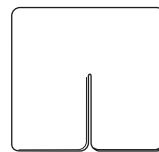
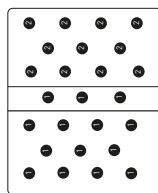
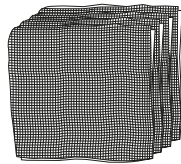
Gebrauchsanweisung User Manual

8020 drainova® Verbandmaterial-Set
8020 drainova® Procedure Pack

1 VE = 10 Stück
1 PU = 10 pieces

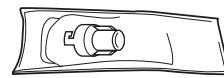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You should have been taught by an ewimed-employee, by the nursing staff or by a doctor how to use the components of this pack. Consult your doctor or the nursing staff if you are having difficulties or questions with applying the procedure pack. Please read this user manual carefully before you start using the drainova® Procedure Pack.

The drainova® Procedure Pack (8020) is hereinafter referred to as „procedure pack“.

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Do you have a question or need any help?

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We will be pleased to answer your questions.

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en

Table of Contents

| | |
|----------------------------|----|
| Product Description | 9 |
| Indication for Use | 9 |
| Procedure Pack Supplies | 9 |
| Warning | 10 |
| Usage of Procedure Pack | 11 |
| Overview Drainage Products | 15 |

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Product Description


The procedure pack contains the necessary supplies for the dressing of a PleurX® catheter or others after performing a drainage procedure.

Intended use

The procedure pack is indicated for dressing the exit side of a PleurX® Pleural or Peritoneal catheter or another catheter.

Procedure Pack Supplies

1 x Self-adhesive
Dressing

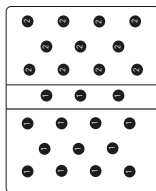
Blue Wrapping
(Wrapped around the following items):


1 Pair of Gloves


1 x White Emergency
Slide Clamp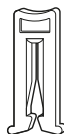


1 x Foam Catheter
Pad

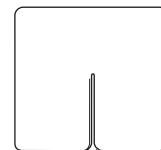1 x drainova® Valve Cap
(9010)

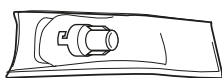4 x Gauze Pads 4" x 4"

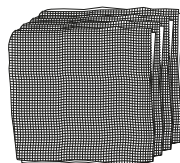3 x Alcohol Pads


Warning

Never use scissors or other sharp objects near the PleurX[®] catheter or other catheter.

If the sealing is not given anymore by the safety valve of the catheter (for example if you accidentally cut the catheter), follow these steps:

- a. Pinch the catheter closed between your fingers.
- b. Slip the white emergency slide clamp over the catheter and push the catheter completely into the small end of the clamp. This will close the catheter.
- c. Inform your doctor immediately.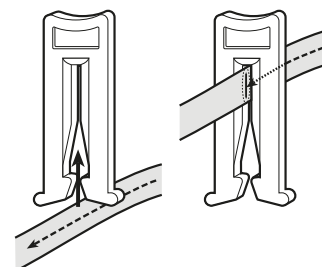

This product is for single use only.

Do not reuse to avoid cross contamination.

The procedure pack has been sterilized.

Sterile technique should be used when changing the dressing.

Do not use if product or package is damaged.

It might be a non-functional or non-sterile product.

Pay attention to the expire date.

Do not use if expire date is exceeded.

Cleaning/desinfection.

If fluid leaks, wash your hands with water and soap and use a suitable cleaning product for the surface.

Pay attention to the user manuals.

When performing a drainage procedure pay attention to the two user manuals, both for PleurX[®] vacuum bottles (50-7510 / 50-7500B or 50-7510/4 / 50-7500B/4) and ewimed gravitational reservoirs (50-7505 or 50-7505/4).

The alcohol pads are easily flammable.

Do not expose the pads to an open flame.

The product is not made with natural latex rubber and does not contain phthalates.

Do not re-sterilize.

ewimed will not accept a credit or exchange for any product that has been opened but not used.


Usage of Procedure Pack

For performing a drainage procedure refer to the instruction for use of the PleurX® Vacuum Bottles (50-7510/50-7500B or 50-7510/4 / 50-7500B/4) or ewimed gravitational reservoirs (50-7505 or 50-7505/4).

If you want to use the procedure pack for dressing the catheter exit side without performing a drainage, follow these instructions:

Note: The valve cap will not be necessary for this part. Do not throw it away and keep it for other purposes.

1. Set up a clean workspace on a table or counter.
Note: For disposal of used items an open container should be available.
2. Thoroughly wash your hands with soap and water for at least 1 minute.
3. Remove the dressing from over the catheter. Be sure not to tug on the catheter: support the skin while removing the self-adhesive dressing, gently grasp one edge and slowly peel the self-adhesive dressing from the skin. Take care not to touch the catheter exit site directly.
Note: Avoid skin trauma by peeling the self-adhesive dressing back rather than pulling it from the skin. If you see any swelling or fluid around the catheter, finish this drainage procedure and contact your doctor or nurse.
4. Again thoroughly wash your hands with soap and water for at least 1 minute.
5. Open the procedure pack pouch. Set the self-adhesive dressing aside.
6. Set bundle with blue wrapping on your workspace with the flap side up. Carefully unfold the blue wrapping by pulling on the outside and on the edge of the wrapping. Leave the enclosed items on the wrapping.

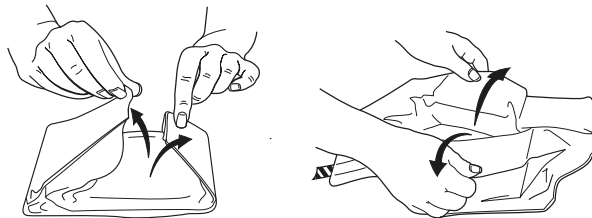
The items and the inside of the wrapping are sterile. Do not touch them with your ungloved hands or other non-sterile items.


7. Pick up gloves at the folded cuff and put them on as shown below. Both gloves fit either hand. Be careful not to touch the outside of the gloves with something non-sterile, such as your skin or clothing.


8. Tear open the three alcohol pads but do not remove the pads from their pouches. Place them on the blue wrapping with some distance to the other sterile components.
9. If you need the new valve cap:
Open the package around the valve cap and let the valve cap fall onto the blue wrapping. Remove the valve cap of the catheter and clean the valve with an alcohol pad. Put the new valve cap on the valve of the catheter. Afterwards dispose the consumed valve cap and the alcohol pad.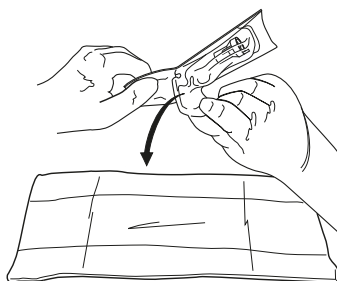


10. Use a new alcohol pad to clean the catheter exit side. Discard the alcohol pad.
Note: Make sure the exit side and skin around the catheter are dry before applying the self-adhesive dressing procedure.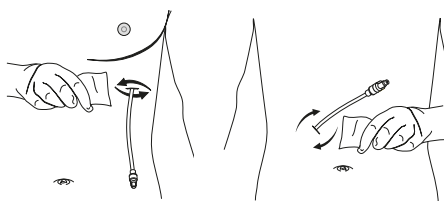


11. Place the foam catheter pad around the catheter.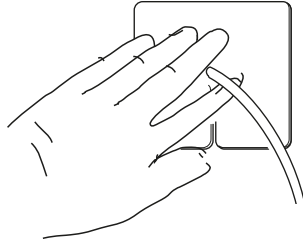

12. Wind the catheter into loops and place it over the foam pad.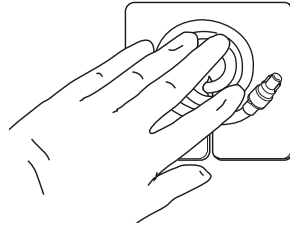

13. Cover the catheter with four gauze pads.

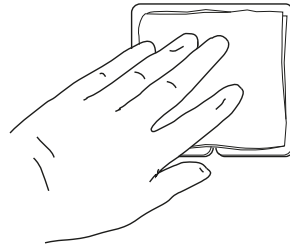14. The self-adhesive dressing has two parts:
a. The self-adhesive side numbered as 1 and 2.
b. The backside numbered as 3.
15. Center the self-adhesive dressing over the gauze pads. The back side (3) should be visible now. Slowly pull off the film 1 from below and push it on the gauze pads.
16. Afterwards pull off the other side (2) from below and stick it on the gauze pads as well.
17. Uncover the backside of the self-adhesive dressing marked as 3 and flatten the edges of the self-adhesive dressing.
18. Press and flatten the self-adhesive dressing from the center towards the edges using firm pressure to enhance adhesion.

Note: Please dispose the used procedure pack per applicable local regulations.

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| Produktbeschreibung | Art.-Nr. | Produktbeschreibung | Art.-Nr. |
|----------------------------------------------------|-----------------|---------------------------------------------------|-----------------|
| PleurX® Drainage-Set (1000 ml Flasche) VE=10 | 50-7510 | ewimed Schwerkraftreservoir (2000 ml) VE=10 | 50-7220 |
| PleurX® Starter Kit (1000 ml Flasche) VE=4 | 50-7510/4 | ewimed Drainage-Set (2000 ml Beutel) VE=10 | 50-7505 |
| PleurX® Drainage-Set (500 ml Flasche) VE=10 | 50-7500B | ewimed Starter Kit (2000 ml Beutel) VE=4 | 50-7505/4 |
| PleurX® Starter Kit (500 ml Flasche) VE=4 | 50-7500B/4 | PleurX® Spülschlauch VE=10 | 50-7245 |
| PleurX® vakuumdichtes Reservoir (1000 ml) VE=10 | 50-7210 | PleurX® Katheter-Schutzkappe VE=10 | 50-7235 |
| ewimed Pneu-Pack I VE=10 | P8531 | ewimed Pneu-Pack II VE=10 | P7899 |
| ewimed Pneu-Pack III VE=10 | P8103 | Sicherheitsklemme für PleurX® VE=10 | P1100 |

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For additional information about our products, please contact us or visit our website at: www.ewimed.com.

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DEHP-frei /
DEHP-free

drainova® is a trademark and/or registered trademark of ewimed.

PleurX® is a trademark and/or registered trademark of CareFusion Corporation, or one of its subsidiaries.

Shaoxing Yibon Medical Co. Ltd.
No. 341 Yuewang Road
Paojiang Industrial Zone
Shaoxing
312000 Zhejiang, CHINA

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EURASIA Deutschland GmbH