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en drainova® Reservoir, soft vakuum

1 Access tip **2** Pinch clamp **3** Drainage Tubing **4** One-way valve* **5** Reservoir **6** Drainage measuring scale **7** Upper stop button **8** White funnel adapter **9** Orange safety catch

(*) Explanation of terms in the glossary

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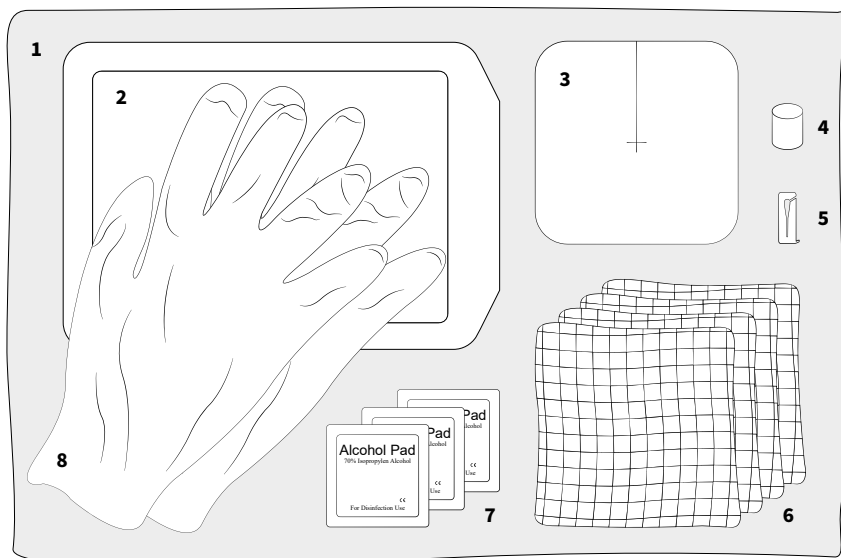
en Dressing set

1 Blue drape 2 Self-adhesive film dressing
3 Foam pad 4 drainova® silicone cap
5 Emergency slide clamp 6 4 × gauze pads
7 3 × alcohol wipes 8 1 pair of sterile gloves

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10 **drainova® Reservoir, Drainage-Set**
Gebrauchsanweisung

de

25 **drainova® Reservoir, Drainage Set**
Instructions for use

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Content

2	Explanation of Symbols
6	Drainage Log Chart
7	Product Description
26	The drainova® Catheter / drainova® ArgentiC Catheter or PleurX™ Catheter
26	The drainova® Reservoir Drainage Set
27	Warnings
	general
	in relation to your catheter
29	Preparation
30	Procedure
33	Final steps
34	Applying a new dressing
36	Emptying the drainova® Reservoir
36	Frequently asked questions
37	Returns/refunds
38	Glossary

en

You should have already been instructed in the correct use of the components in this set either by ewimed training staff, district nursing staff or a doctor. Please contact your doctor or a member of the nursing staff if you are experiencing difficulties or have questions relating to effusion drainage.

Please carefully read the instructions before using the drainova® Reservoir Drainage Set.

You have questions or need help?

Contact us:

DE: +49 7471 73972-0
info@ewimed.com

AT: +43 2231 22500
info@ewimed.com

EN: +49 7471 73972-0
info@ewimed.com

SE: +46 8 25 11 69
info@fenik.se

CH: +41 52 577 02 55
info@fenik.ch

HU: +36 1 998 8295
info@ewimed.hu

For further detailed information please visit our website
www.ewimed.com

The drainova® Catheter / drainova® ArgentiC Catheter or PleurX™ Catheter

The drainova® Catheter, drainova® ArgentiC Catheter or PleurX™ Catheter serve to drain fluid from the chest cavity (pleural effusion*) or the peritoneal cavity (ascites*).

Your doctor has implanted this catheter so that you can independently drain the pleural effusion* or the ascites* at home according to your needs - rendering repeated visits to hospital for puncturing unnecessary.

The implanted part of the catheter is perforated with drainage holes, which allow the effusion to drain out. During implantation, the polyester cuff is placed in a subcutaneous tunnel to prevent infections*. The safety valve on the visible part of the catheter is generally closed, preventing both leakage of effusion and air from entering.

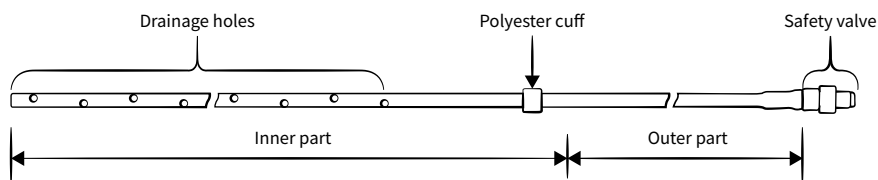
Illustration of a drainova® catheter

A protective cap or a drainova® silicone cap is attached to the safety valve to protect it from contamination or damage when not draining.

To drain effusion fluids use the drainova® Drainage Set or the ewimed Drainage Set. The key-lock-principle ensures the optimal connection of the drainage sets with the safety valve.

The drainova® Catheter, drainova® ArgentiC Catheter or PleurX™ Catheter will hereinafter be referred to as "catheter".

The drainova® Reservoir Drainage Set

The drainova® Reservoir Drainage Set contains sterile material required for the safe and easy drainage of effusion fluids with your catheter.

The drainova® Reservoir Drainage Set will hereinafter be referred to as Drainage Set.

The container found in the drainova® Reservoir Drainage Set will hereinafter be referred to as "drainova® Reservoir".

All components contained in the set are CE labelled and have been sterilized with ethylene oxide (see individual packaging).

We recommend the following additional items for carrying out the drainage:

- Alcoholic disinfection spray
- Hand disinfectant
- Non-sterile disposable gloves

Warnings

general

Do not use this product if:

- the sterile packaging is damaged
- damaged
- the expiry date has elapsed

The drainage consumables are intended for single use only!

The drainage sets are delivered in sterile packaging which have undergone quality controls. The drainage sets can not be re-sterilised.

The product is intended for single use only. Re-use could contribute to a cross-contamination.

ewimed GmbH accepts no liability for re-used, re-sterilised products or products taken from damaged packaging.

Drainage of the chest cavity

Only drain the amount of fluid prescribed by your doctor.

ewimed does not recommend draining more than 1000 ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in order to pause the flow.

Possible complications relating to repeated drainage of pleural effusions* include pneumothorax*, re-expansion pulmonary oedema*, hypotension*, circulatory collapse*, septa* in the chest cavity or infection*.

Drainage of the peritoneal cavity

Only drain the amount of fluid prescribed by your doctor.

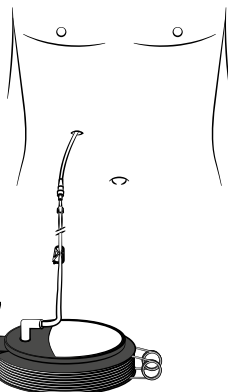
ewimed does not recommend draining more than 2000 ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in order to pause the flow.

Possible complications relating to repeated drainage of ascites* include hypotension*, circulatory collapse*, electrolyte imbalance*, loss of protein*, hepatorenal syndrome*, leakage of ascites from incision sites, peritonitis*, septa* in the peritoneal cavity* and infection.

Position of the drainova® Reservoir during drainage

The drainage of fluid is assisted by the force of gravity. Therefore please ensure that the drainova® Reservoir is always positioned under the catheter exit site whilst draining, e.g. on the floor.

During drainage, make sure that no pressure is applied to the reservoir. This could damage the one-way valve and cause contamination from the backflowing fluid.

Cleaning / disinfection

Should fluid leak out and come into contact with skin, clean the skin with soap and water. Use suitable cleaning or disinfectant products for surfaces. Do not use Octenisept or products containing iodine to clean/disinfect the catheter or safety valve.

The alcohol wipes are slightly flammable.

Keep the wipes away from naked flames.

Warnings

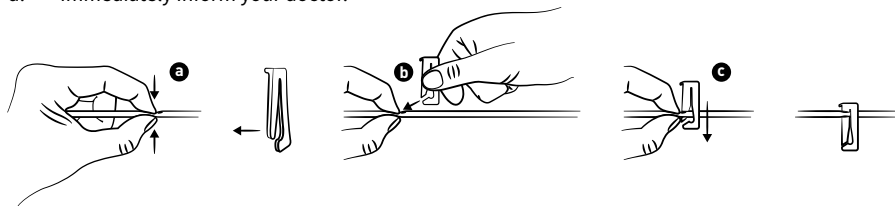
in relation to your catheter

Never cut through the catheter or cut the safety valve off.

Ensure that the catheter does not come into contact with any sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i.e. is no longer attached), proceed as follows:

- firmly pinch the catheter closed with your fingers
- using your other hand, hold the emergency slide clamp open so that the catheter can be fed through the open end of the clamp.
- push the catheter completely through the narrow end of the clamp.
- Immediately inform your doctor.

Only use original accessories.

Do not attempt to connect the catheter to non-original drainage products. Only the specially-developed access tip can be inserted into the safety valve. The use of foreign items can damage the safety valve. Incorrect handling can cause air to enter the body or effusion fluid to leak out.

Always keep the catheter safety valve clean and the access tip on the connected tube sterile.

Take care to ensure that they do not come into contact with any un-sterile objects in order to avoid contamination. The safety valve must be disinfected before each drainage.

While draining, make sure that the safety valve and the access tip are securely connected. Take care not to tug or pull on the connected tube.

"Take particular care to ensure that the access tip is inserted in the safety valve in a sterile manner and make sure that the connection is secure. It can lead to contamination if the parts become disconnected by mistake. In case this occurs, disinfect the safety valve with a new alcohol wipe and dispose of the disconnected drainage material. Use a new sterile drainage kit to continue with the drainage. The safety clamp can be used to prevent unintentional disconnection."

The safety valve is not contained in the drainage set but can be prescribed separately.

en

Preparation

The effusion fluid should be drained using a drainage set according to the prescription filled out by your doctor. If you have not received a medical recommendation for the drainage interval or volume, you will receive one during training.

1. Prepare a clean, empty work space on a table or other work surface.

Please note: Place an open container nearby to dispose of used material.

2. Disinfect your hands. We also recommend washing your hands thoroughly before disinfecting.
3. Put on the non-sterile disposable gloves.
4. Remove the dressing covering the catheter. Carefully peel away a corner of the dressing and pull it slowly away from the skin, making sure that the catheter is not accidentally pulled out.

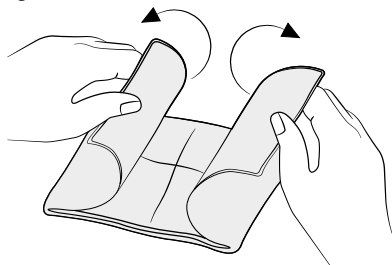
Important: Inform your doctor should you notice swelling, signs of inflammation or an accumulation of fluid at the catheter exit site or incision site.

5. Take off your gloves and disinfect your hands again.
6. Open the packaging of the drainova® Reservoir Drainage Set.
7. Open the dressing set by pulling both sides of the packaging at the top apart. Place the self-adhesive film dressing to the side.


Procedure

1. Place the dressing set with the flap facing up on the work surface. Carefully unfold the blue drape. Leave the contained items on the drape.

The items in the dressing set are sterile and must not come into contact with non-sterile objects.

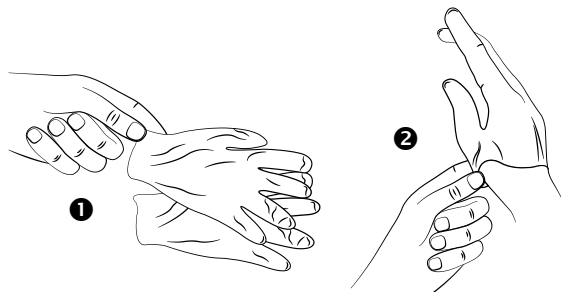2. Pick up the gloves at the opening and put them on as illustrated. Both gloves are suitable for either hand. Make sure that the gloves do not come into contact with non-sterile objects e.g. skin or clothing.

- 2.1. Tear open the alcohol wipe sachet but do not take the wipe out. Open the sachets so that these can be placed on the safety valve later on. Place the opened alcohol wipe sachets back on the edge of the blue drape.
3. Remove the drainova® Silicone Cap or the PleurX™ Protective Cap:

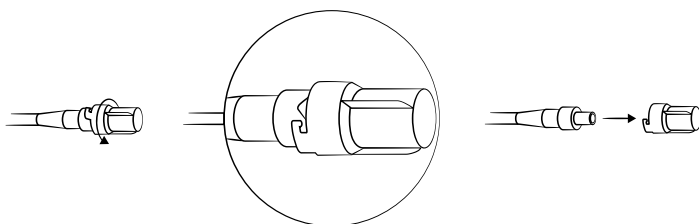
- **Using the drainova® Silicone Cap:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® Silicone Cap. Dispose of the cap.


- **Using the PleurX™ Protective Cap:**

Firmly hold the safety valve of the catheter, rotate the PleurX™ Protective Cap anticlockwise and carefully remove. Dispose of the cap.

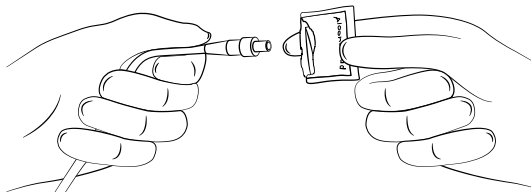
4. Insert the safety valve into one of the open alcohol wipe sachets and clean the valve using gentle circular movements. Leave the safety valve in the sachet whilst carrying out steps 5 - 7.


Warning: Do not use Octenisept® or other disinfectants containing iodine.

en

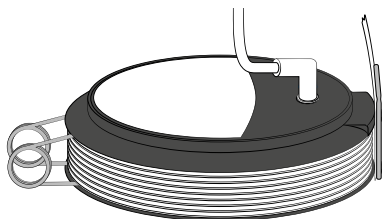5. Open the packaging of the drainova® Reservoir and take it out.
The product in the packaging is sterile. It is important that the access tip on the tubing remains sterile. Place the drainova® Reservoir next to the blue drape.
Remove the tape wrapped around the coiled up tubing.

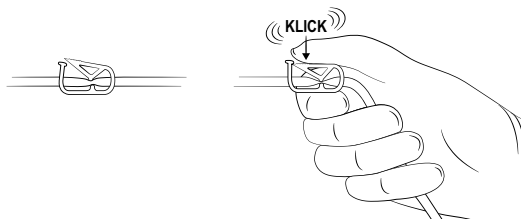
6. Hold the base of the access tip in one hand. Completely close the pinch clamp on the tube with the other hand.


7. Continue to hold the access tip at the base and carefully remove the protective film from the access tip (if necessary by gently rotating).

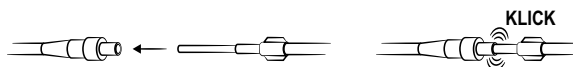- 8.1 Remove the alcohol wipe sachet from the safety valve and insert the access tip in the safety valve. Ensure that the safety valve and the access tip are securely connected during drainage. A click can be heard and felt when connected.

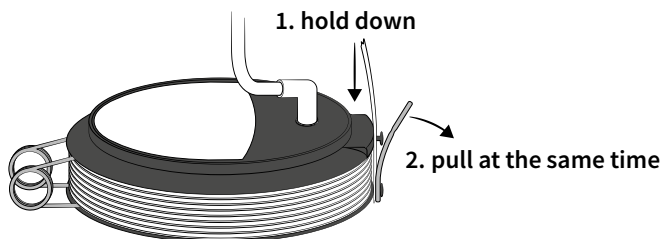
8. Push down on the drainova® Reservoir to remove the orange-coloured safety clasp. Keep the orange-coloured clasp for later disposal of the reservoir.

Recommendation: The duration of drainage can be shortened by placing the drainova® Reservoir at a lower height e.g. on the floor.

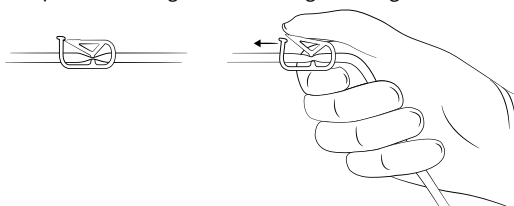9. Open the pinch clamp on the tubing so that drainage can begin.


Please note: The flow can be slowed or interrupted when:

- The fluid has almost completely drained
- The fluid has a high viscosity
- The safety valve is blocked due to a clot* or a protein thread


You can find additional information on how to proceed in such a situation in the FAQ-section from page „Frequently asked questions“ auf Seite 36 onwards.

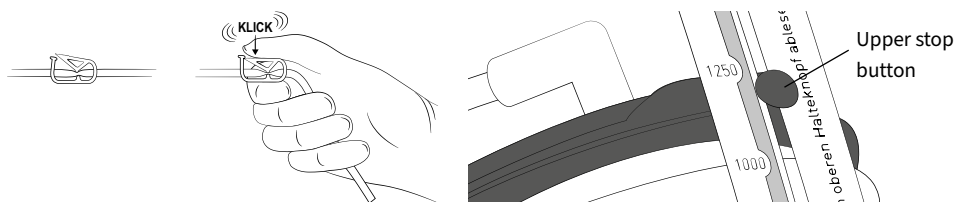10. When you have reached the drainage amount recommended by your doctor, stop drainage by closing the pinch clamp. The top stop button shows you the drained amount. You can record the amount in the drainage log chart on page 4.


Final steps

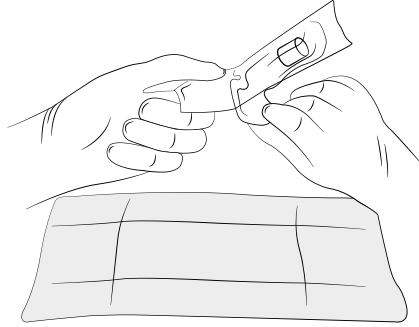1. Open the packaging of the drainova® Protective Cap and let the cap fall onto the blue drape.

en

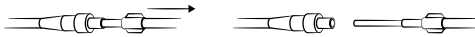
2. Take the tubing in one hand and the safety valve in the other hand. Pull the access tip in a firm, smooth motion out of the safety valve. Place the tubing down.

3. Clean the safety valve for approx. 30 seconds with an alcohol wipe or other suitable alcohol-based disinfectant and then dispose of the used wipe.


Warning: Do not use Octenisept® or other disinfectants containing iodine.

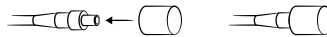4. Attach either the drainova® Silicone Cap or drainova® Protective Cap:
Securely attach the new sterile Drainova® Silicone Cap onto the safety valve of the catheter. The cap will stay in place without the need for additional support.


Applying a new dressing

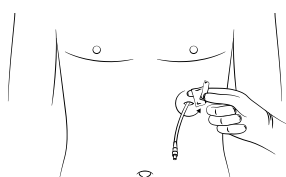1. Clean the area around the catheter exit site with the skin disinfectant spray containing alcohol. If the wound has healed and the sutures have been removed, you can use an alcohol wipe. Please ensure that after applying the disinfectant, the catheter exit site and the skin surrounding the catheter is dry. Only then should you apply the new dressing.

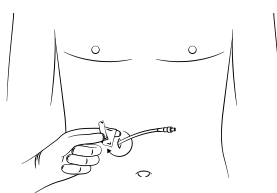
Disinfecting the pleural
catheter exit site


Disinfecting the ascites*
catheter exit site

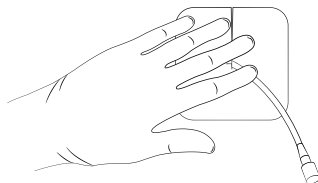
2. Place the foam pad with the slit facing upwards around the catheter.

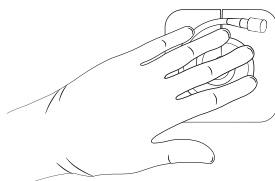3. Loop up the catheter and place it on the foam pad.

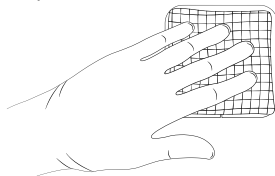
4. Cover the catheter with the gauze pad.


5. Take off both gloves.
6. The self-adhesive film dressing is comprised of three layers.
 - I. Non-printed protective film (two pieces)
 - II. Self-adhesive film dressing (transparent)
 - III. Printed protective film

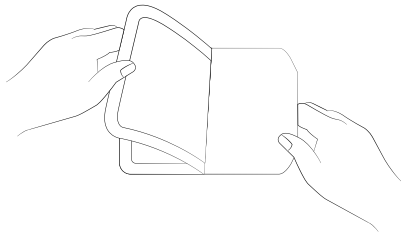
7. Expose the self-adhesive surface by peeling off the printed film.

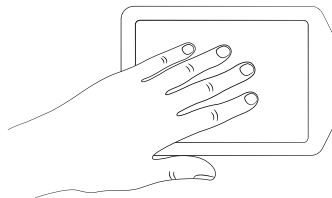8. Centre the self-adhesive film dressing over the gauze pads and attach to the surrounding skin.

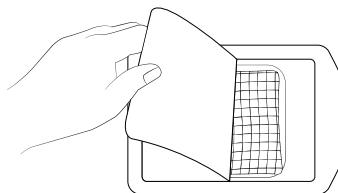
9. Detach the middle part of the non-printed protective film from the film dressing.

10. Carefully peel the edge of the non-printed protective film away from the film dressing and smooth out the entire self-adhesive film dressing starting from the centre to the edges.

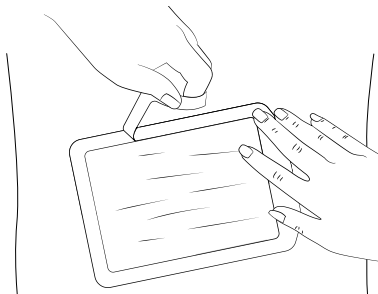Please note: Dispose of the used dressing according to the local regulations.

Emptying the drainova® Reservoir

1. For the following steps you will need scissors, a towel, the orange-coloured safety clasp, a plastic bag (e.g. the outer packaging in which the drainova® Reservoir came in).
2. Place the drainova® Reservoir on the floor. Open the pinch clamp and lift up the tube. Residual fluid in the tube can now drain off.
3. Close the pinch clamp.
4. Now cut through the tube below the one-way valve* (Fig.1). Dispose of the cut-off piece of tube with the pinch clamp and access tip. If necessary, use the laid out cloth. To dispose of the collected fluid in the toilet, position the reservoir upside down so that the tube opening is pointing downwards above the toilet. You can start the emptying process by compressing the reservoir.

Warning: Used products and effusion fluid may pose biological risks. Therefore please observe the local and national regulations when disposing of used products and fluid.

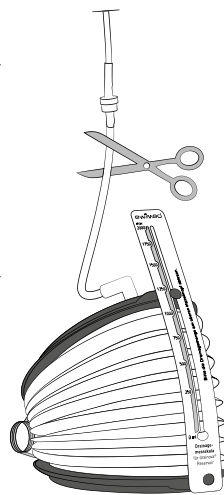
illustration 1

5. Once the reservoir is emptied and compressed, attach the orange safety catch. Make sure that, where possible, there is no remaining fluid in the tube. Put the reservoir together with the drainage tubing into the plastic bag provided.
6. Dispose of the bag in accordance with the local regulations.

Frequently asked questions

When can the sutures (stiches) be removed?

The sutures can typically be removed after 7 – 10 days. The suture holding the catheter in place however, should not be removed before 30 days have elapsed.

How often do I need to drain?

Only drain the amount prescribed by your doctor.

Fluid should be drained every one to two days (or as prescribed by your doctor) using a drainage set. Please consult your doctor if you wish to drain more or less often.

ewimed does not recommend draining more than 1000 ml from the pleural cavity.

ewimed does not recommend draining more than 2000 ml from the peritoneal cavity.

What should I do if I am still short of breath after draining?

Please inform your doctor if, after successfully draining fluid out of the pleural or peritoneal cavity, you still have difficulty breathing.

What should I do if the volume of fluid becomes less and less or if no fluid at all can be drained?

There can be two possible reasons:

1. If there is a significant difference in volume compared to the last drainage output or if no fluid can be drained at all, it is possible that the catheter is blocked or a protein thread is in the access tip hindering drainage. Try to dislodge the blockage by squeezing and rubbing along the catheter. If this does not help, please consult your doctor or contact the respective company (please refer to the contact details on page 25).
2. If the drainage output continually decreases and you have no symptoms, discomfort or weight gain, it is possible that there is no longer fluid being produced. Please consult with your doctor on how to proceed.

Can I shower or have a bath with the catheter?

You can shower as long as the catheter is completely and correctly covered with the self-adhesive film dressing. If the gauze pads however do get wet during showering, remove the dressing, clean and dry the area and apply a new dressing. The catheter should not be completely immersed in water either in the bath or in the swimming pool. Avoid direct contact of the safety valve with bath/shower water.

Please note: We recommend showering before draining or applying a new dressing, so that the dressing can be promptly changed should it become wet.

What should I do if I accidentally pull out the Catheter?

The Catheter should have grown together with the subcutaneous skin tissue through the polyester cuff. If the catheter is accidentally pulled out and the cuff is visible, cover the exit site with a sterile dressing and immediately contact the clinic that carried out the implantation.

How can I tell if I have an infection?

Immediately inform a doctor should you experience pain, skin redness (erythema*), if the skin is warm to the touch, swelling (oedema*), fever or accumulation of fluid as these can be signs of infection*.

Should you notice a change in colour of the effusion fluid, proceed as described.

Shortly after implantation redness and mild pain can be expected. These symptoms should not last or become worse.

Returns/refunds

Only originally packaged (unopened, non-damaged, non-labelled) boxes of drainage sets will be accepted for return and refund.

Glossary

Ascites	Accumulation of fluid in the abdomen
Circulatory collapse	Temporary loss of consciousness
Clot	Blood clot
DEHP	Softening agents in plastics
Electrolyte imbalance	Imbalance of ions (e.g. natrium, potassium, calcium) in the blood or somatic cells
Erythema	Skin redness
Hepatorenal syndrome	Impaired renal function
Hypotension	Low blood pressure
Incision	Skin incision during the operation e.g. for the implantation of the catheter
Infection	Reaction of the body's own defence mechanism to germs e.g. bacteria, viruses etc.
Loss of protein	Reduction in the amount of protein in the blood (protein depletion)
Oedema	Swelling due to excessive accumulation of fluid
One-way valve	Non-return system for increased patient safety
Peritoneal cavity	Part of the abdominal cavity
Peritonitis	Infection of the peritoneum
Phthalate	Softening agents (e.g. DEHP)
Pleural effusion	Effusion (accumulation of fluid) in the pleural cavity
Pneumothorax	Air in the area between the lung and the chest wall
Re-expansion - Pulmonary oedema	Fluid retention in functional lung tissue due to sudden expansion of the previously compressed lung; acute shortness of breath
Septa	Tissue bridging / building of chambers between separated areas in a body cavity (e.g. abdomen)
Subcutaneous tissue	Tissue beneath the skin

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PleurX™ is a trademark and /or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and /or registered trademark of ewimed.
Shaoxing Yibon Medical Co., Ltd.
OBELIS S.A.
EURASIA Deutschland GmbH
Germany | ewimed GmbH | Dorfstrasse 24 | 72379 Hechingen | Tel.: +49 7471 73972-0 | info@ewimed.com