**de ewimed Drainage-Set, 2000 ml**

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**en ewimed Drainage Kit, 2000 ml**

Instructions for use

**fr ewimed kit de drainage, 2000 ml**

Instructions

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
istruzioni d'uso

**sv ewimed dränageset, 2000 ml**

Bruksanvisning

ewimed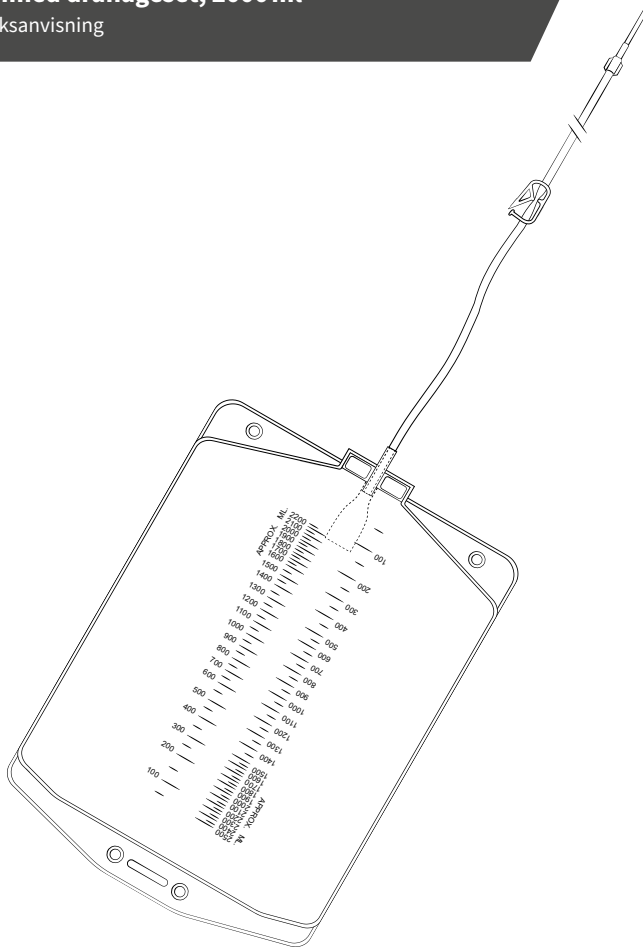

**REF**

50-7505, 50-7505/4

# de Erklärung der Symbole

## en Explanation of Symbols

### fr Explication des symboles

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**de** Bitte beachten Sie, dass nicht alle dargestellten Symbole auf Ihr Produkt zutreffen. Die jeweilig zutreffenden Symbole finden Sie auf dem Etikett Ihres Produktes. / **en** Please note that not all symbols shown apply to your product. The relevant symbols can be found on the product label. / **fr** Veuillez noter que tous les symboles décrits ne s'appliquent pas à votre produit. Vous trouverez les symboles correspondants sur l'étiquette de votre produit. / **it** Si prega di notare che non tutti i simboli mostrati sono applicabili al vostro prodotto. Troverete i simboli pertinenti sull'etichetta del vostro prodotto. / **sv** Observera att inte alla avbildade symboler är tillämpliga för din produkt. Vilka symboler som gäller för din produkt hittar du på produktetiketten.

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# Content

- 2 Explanation of Symbols
- 5 Drainage Log Chart
- 6 Products
- 25 The drainova® Catheter, drainova® ArgentiC Catheter or PleurX™ Catheter
- 25 The ewimed Drainage Kit
- 26 Warnings
  - general
  - in combination with the catheter
- 28 Preparation
- 29 Drainage Procedure
- 31 Final steps
- 32 Applying a new dressing
- 34 Emptying the Reservoir
- 34 Frequently Asked Questions
- 36 Returns/ Refunds
- 36 Glossary

en

You should have already been instructed in the correct use of the components contained in this set by our training staff, nursing staff or a doctor. Please consult your doctor or a member of nursing staff if you experience any difficulties or have questions regarding the drainage of effusion fluid.

**Thoroughly read the entire instruction manual before using the ewimed Drainage Kit to drain the effusion.**

## You have questions or need help?

Contact us:

DE: +49 7471 73972-0  
info@ewimed.com

AT: +43 2231 22500  
info@ewimed.com

EN: +49 7471 73972-0  
info@ewimed.com

SE: +46 8 25 11 69  
info@fenik.se

CH: +41 52 577 02 55  
info@fenik.ch

**For further detailed information please visit our website  
[www.ewimed.com](http://www.ewimed.com)**

# The drainova® Catheter, drainova® ArgentiC Catheter or PleurX™ Catheter

The drainova® Catheter/drainova® ArgentiC Catheter or PleurX™ Catheter serves to drain effusion fluid from the chest cavity (pleural effusion\*) or the abdominal cavity (ascites\*).

Your doctor has implanted this catheter so that you can independently drain pleural effusion\* or ascites\* at home as required, saving you the need for repeated trips to the hospital or clinic for punctures.

The part of the catheter implanted in the body features drainage holes through which the effusion can be drained out. During implantation, the polyester cuff is placed in a tunnel under the skin to protect against infection\*. The safety valve on the visible part of the catheter is generally closed to prevent air from entering and leakage of effusion.

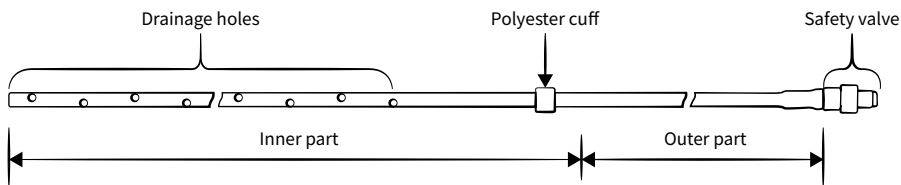
Illustration: Schematic representation of a drainova® catheter

When not draining, a protective cap is fitted onto the safety valve or a drainova® silicone cap is placed on to the safety valve to protect against contamination or damage.

Use the drainova® Drainage Set or the ewimed Drainage Kit to drain the effusion. The drainage kit, featuring the key-lock principle, is specifically designed for connection with the safety valve .

Hereinafter the drainova® Catheter, drainova® ArgentiC Catheter or PleurX™ Catheter will be referred to as "catheter".

## The ewimed Drainage Kit

The ewimed Drainage Kit contains sterile materials that you will need for the safe and easy drainage of effusion with your catheter.

The gravity reservoir contained in the ewimed Drainage Kit will hereinafter be referred to as the "reservoir".

All products are CE labelled and have been sterilized with ethylene oxide gas (see individual packaging).

### For the drainage procedure, we additionally recommend:

- Alcohol-based skin disinfectant spray
- Hand disinfectant
- Non-sterile disposable gloves

# Warnings

## general

### **Do not use this product if:**

- The sterile packaging is damaged
- The product is damaged
- The expiry date has expired

### **The consumables are intended for single use only!**

The ewimed Drainage Kit is delivered in a sterile condition, which is ensured by appropriate quality controls. The Drainage Kit must not be resterilised.

The product is for single-use only. Re-use can lead to cross-contamination.

ewimed GmbH accepts no liability for products that have been re-used, resterilised or removed from damaged packaging.

### **Drainage from the chest cavity**

Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 1000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage. Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of pleural effusion\* include pneumothorax\*, re-expansion pulmonary edema\*, hypotension\*, circulatory collapse\*, septa\* in the chest cavity or infection\*.

### **Peritoneal drainage**

Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 2000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage. Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of ascites\* include hypotension\*, circulatory collapse\*, electrolyte imbalance\*, protein loss\*, hepatorenal syndrome\*, leakage of ascites\* through the incisions\*, peritonitis\*, septa\* in the peritoneal cavity\* and infection\*.

### **Position of the reservoir during drainage**

As the effusion is drained by gravity, make sure that the reservoir is always situated below the catheter exit site during drainage.

## Cleaning / Disinfection

If effusion is leaking out and comes in contact with skin, clean the skin with soap and water. For surfaces, use suitable cleaning products or surface disinfectant. Do not use Octenisept or disinfectants containing iodine for cleaning/disinfecting the catheter or safety valve.

### The alcohol wipes are highly flammable.

Keep the wipes away from naked flames.

## Warnings


in combination with the catheter

### Never cut through the catheter or cut the safety valve off

Avoid contact of sharp objects with the catheter.

If the function of the safety valve can no longer be guaranteed or if the safety valve has been accidentally cut off or is no longer present, proceed as follows:

- Firmly squeeze the catheter between your fingers.
- With your other hand, hold the emergency slide clamp open so that the catheter can be inserted.
- Push the catheter completely into the narrow end of the clamp.
- Immediately contact the clinic where the implantation took place.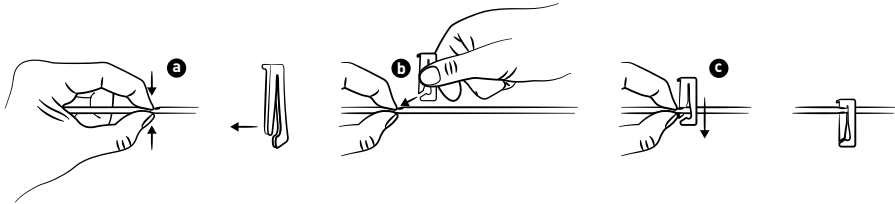


### Use only original accessories.

Only original accessories may be connected to the catheter. Do not insert anything other than the specially-designed access tip into the safety valve, otherwise the safety valve may become damaged. Improper handling can cause air to enter the body or fluid to permanently leak out.

### Keep the safety valve on the catheter clean and the access tip on the connection tube sterile.

Ensure that they do not come into contact with non-sterile objects to avoid contamination. The safety valve must be disinfected before each drainage.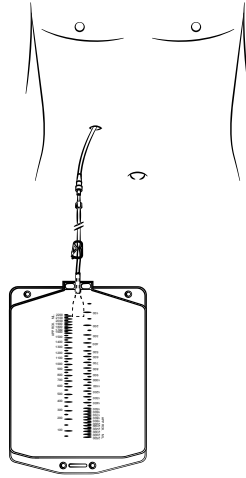

## When draining, ensure that the access tip and the safety valve are securely connected. Make sure that the connection tube is not dragged or pulled.


Connect the access tip to the safety valve in a sterile manner and make sure that they are firmly connected. Accidental disconnection may result in contamination. If this happens, disinfect the safety valve using a new alcohol wipe and discard the disconnected drainage material. A new sterile Drainage Kit must be used to continue drainage.

Please observe the local, regional and national waste disposal regulations.

**Please note:** If required (for additional protection against accidental disconnection) the connection can be secured with the separately available drainova® clickFix (Art. No. P1100) .

en

The drainova® clickFix can be additionally prescribed. It is not contained in the Drainage Kit.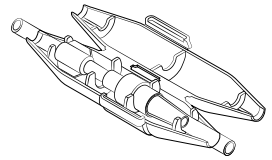

## Preparation

The effusion should be drained with an ewimed Drainage Kit according to your doctor's instructions. If you have not received a medical recommendation for the drainage interval or drainage volume, you will receive a recommendation during training.

1. Prepare a clean, clear work area on a table or other work surface.

**Please note:** When disposing of used material, please observe the local, state and national regulations . If necessary, place a waste container ready.

2. Disinfect your hands. We also recommend washing your hands thoroughly beforehand.
3. Put on the non-sterile disposable gloves.
4. Remove the dressing covering the catheter. Carefully lift off one corner of the dressing and slowly pull away from the skin. Make sure not to accidentally pull on the catheter.

**Important:** Inform your doctor should you notice swelling, signs of inflammation or an accumulation of fluid at the catheter exit site or incision site.


5. Take off the gloves and disinfect your hands again.
6. Open the sterile packaging of the ewimed Drainage Kit.
7. Open the packaging of the dressing-material set by pulling apart both sides of the bag at the top. Set aside the self-adhesive foil dressing.


# Drainage Procedure

1. Place the dressing set with the flap facing upwards. Carefully unfold the blue drape. Place the items contained in the set on the drape.

The items in the dressing set are sterile and must not come into contact with non-sterile objects.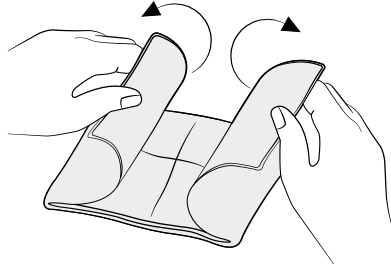

2. Hold the gloves by the opening and put them on as shown. Both gloves are suitable for both hands. Make sure that the gloves do not come into contact with non-sterile objects such as skin or clothing.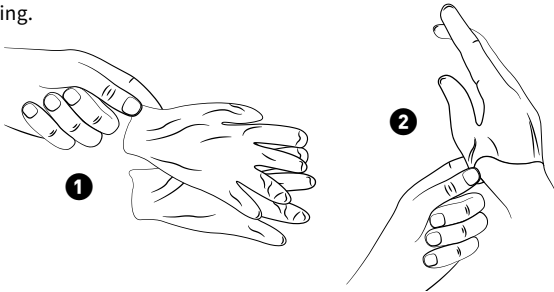

- 2.1 Tear open the alcohol wipe sachets on one side. Do not remove the alcohol wipes from the sachets. Open each sachet in preparation to place them onto the safety valve later on and place the sachets back on the edge of the blue drape.

3. Loosen the drainova® silicone cap or the PleurX™/drainova® protective cap:


- **When using the drainova® silicone cap:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® silicone cap. Dispose of the cap.


- **When using the PleurX™/drainova® protective cap:**

Firmly hold the safety valve of the catheter, turn the PleurX™/drainova® protective cap anticlockwise and carefully remove. Dispose of the cap.

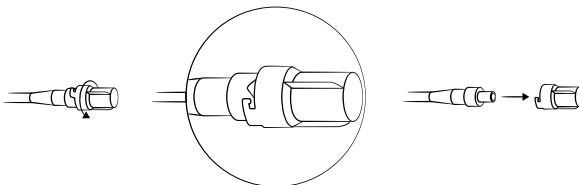
4. Insert the safety valve into one of the already opened alcohol wipe sachets. Gently rotate to clean the safety valve. Leave the safety valve in the sachet while carrying out steps 5 + 6.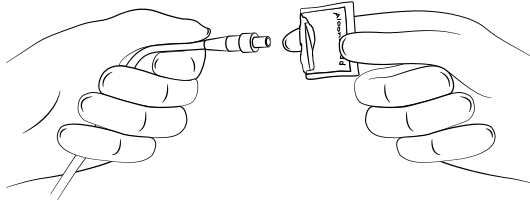

**Important:** Do not use Octenisept® or disinfectants containing iodine.


5. Open the packaging and take the reservoir out.  
The reservoir in the packaging is sterile. It is important that the access tip on the connection tube remains sterile. Place the reservoir next to the blue drape and hold the access tip at the base in one hand. Completely close the pinch clamp on the connection tube with your other hand.

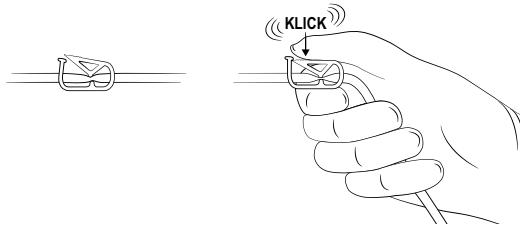
6. Whilst continuing to hold the access tip at the base, carefully pull the protective cover off the access tip and discard.

- 6.1 Remove the alcohol wipe sachet from the safety valve and insert the access tip into the safety valve. Make sure that the safety valve and the access tip are firmly connected. You can hear and feel it click into place.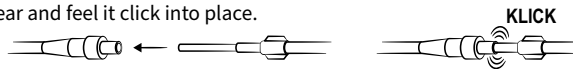

7. The reservoir should always be kept underneath the catheter exit site, as drainage is aided by gravity.
8. Open the pinch clamp on the connection tube so that drainage can begin.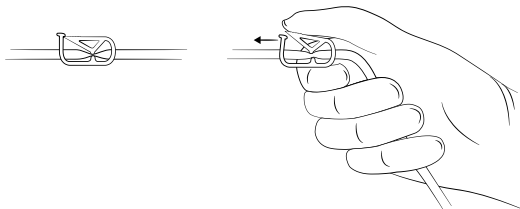


**Please note:** Flow can be slowed or interrupted if:

- The effusion is almost completely drained
- The effusion has a high viscosity
- The safety valve is blocked by a cogel\* or protein thread

Additional information on how to proceed in such a case can be found in the "Frequently Asked Questions" section starting on page 34.


9. When the drainage volume recommended by the doctor has been reached, stop drainage by closing the pinch clamp. Make note of the drainage volume in the "Drainage Log Chart" on page 4.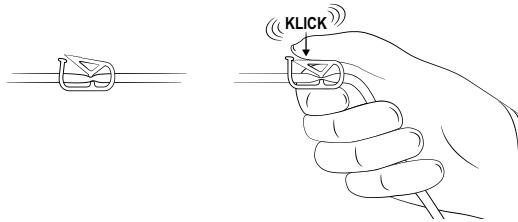


en

## Final steps

1. Open the drainova® silicone cap packaging and let the cap drop onto the blue drape.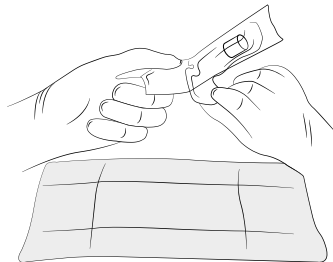


2. Take the connection tube in one hand and the safety valve in the other hand. Pull the access tip out of the safety valve in one firm, smooth motion. Set down the connection tube.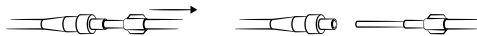

3. Clean the safety valve with an alcohol wipe for about 30 seconds and then dispose of the wipe.


**Important:** Do not use Octenisept® or disinfectants containing iodine.

4. Attach the new sterile drainova® silicone cap firmly onto the safety valve of the catheter. The cap will remain in place without the need for additional fixation.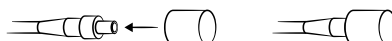


## Applying a new dressing


1. Clean the area around the catheter exit site with the alcohol-based skin disinfectant spray. If the wound has healed and the stitches have been removed, an alcohol wipe may be used. After allowing time for the disinfectant to take effect, please make sure that the catheter exit site and the skin surrounding the catheter are completely dry before applying the new dressing.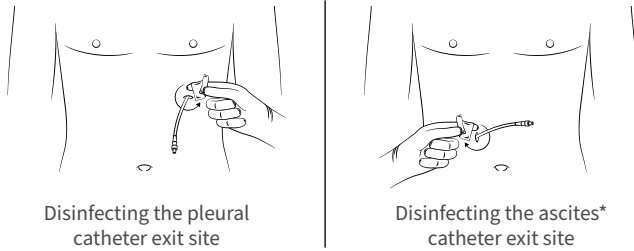


2. Place the foam pad around the catheter with the slit facing upwards.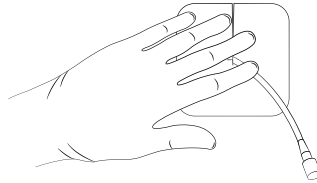


3. Loop the catheter and position it on the foam pad.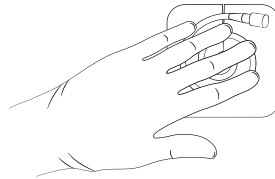


4. Cover the catheter with the gauze pads.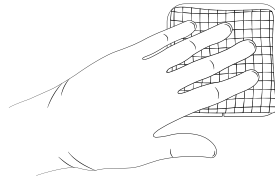


5. Take off both gloves.
6. The self-adhesive film dressing is comprised of three layers.
  - I. Non-printed protective film (two pieces)
  - II. Self-adhesive film dressing (transparent)
  - III. Printed protective film
7. Expose the self-adhesive surface by peeling off the printed protective film.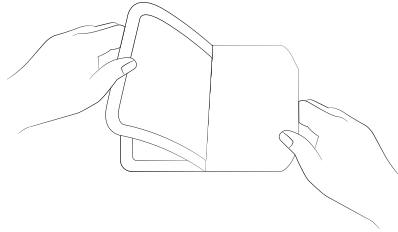

8. Centre the self-adhesive film dressing over the gauze pads and attach to the surrounding skin.

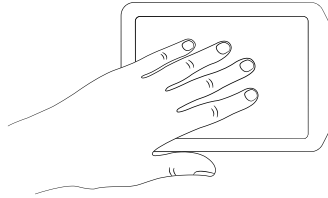
9. Detach the middle part of the non-printed protective film from the film dressing.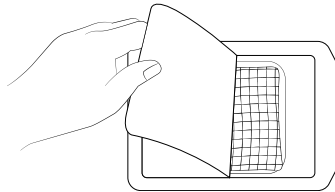


10. Carefully peel the edge of the non-printed protective film away from the film dressing and smooth out the entire self-adhesive film dressing starting from the centre to the edges.

**Please note:** Dispose of used dressing material in accordance with local regulations.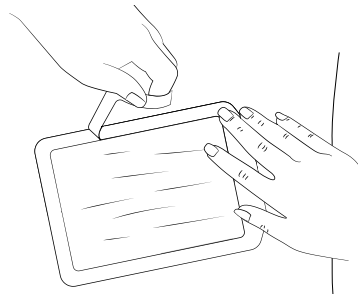


## Emptying the Reservoir

1. Cut off a corner of the reservoir using scissors and empty the contents into the toilet.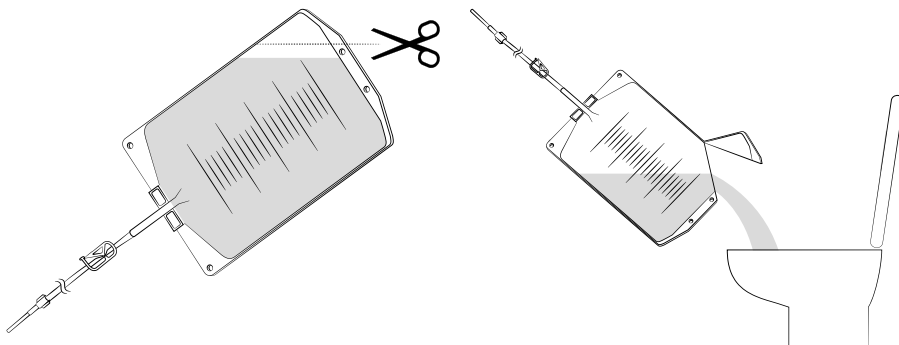

**Please note:** Used products and effusion fluid may pose biological risks. Therefore, please observe local and national regulations when disposing of used products and effusion.

2. Place the reservoir in a plastic bag, seal it firmly and dispose of the bag in accordance with local regulations.

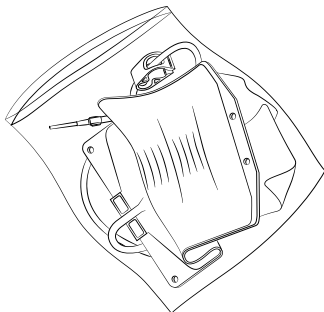## Frequently Asked Questions

### When can the sutures (stiches) be removed?

The incision sutures can typically be removed after 7-10 days. The suture holding the catheter in place however, should not be removed before 30 days have elapsed.

### How frequently should I drain?

Follow your doctor's instructions regarding the drainage quantity.

The effusion fluid should be drained using a Drainage Kit every one or two days or as instructed by the doctor. Consult your doctor if you want to drain more or less frequently.

ewimed recommends draining a maximum of 1000 ml of effusion from the chest cavity.  
ewimed recommends draining a maximum of 2000 ml of effusion from the abdominal cavity.

### **What should I do if I still feel short of breath after draining?**

If you still have difficulty breathing after successfully draining effusion fluid from the chest or abdominal cavity, inform your doctor.

### **What should be done if the effusion volume decreases or no effusion can be drained at all?**

There can be two reasons for this:

1. If the amount of fluid during the last drainage deviates greatly from the usual amount or if no effusion at all can be drained, the catheter may be blocked or a protein thread may be obstructing the access tip, preventing drainage. Try to remove the blockage by squeezing and using stroking movements along the catheter.  
If you are unable to remove the blockage, consult your doctor or contact the company central office (see contact details on page 8).
2. If the drainage amounts continuously decrease and you have no complaints or weight gain, it is possible that no effusion has formed. Discuss next steps with your doctor.

### **Can I shower or take a bath with the catheter?**

If the catheter is completely and correctly covered with the self-adhesive dressing, you can shower. If however, the gauze pads get wet when showering, remove the dressing, clean and dry the area and apply a new dressing. The catheter should not be immersed under water in a bathtub or swimming pool. Avoid direct contact of the safety valve with bath or shower water.

**Please note:** We recommend showering before drainage / changing the dressing, so that the dressing can be promptly changed if it gets wet.

### **What should I do if I accidentally pull out the catheter?**

The catheter should be adhered to the subcutaneous\* skin tissue by the polyester cuff. If the catheter is accidentally pulled out and the cuff is visible, cover the catheter exit site with a sterile dressing and immediately contact the clinic where the implantation took place.

### **How can I tell if I have an infection\*?**

Immediately inform a doctor if you experience pain, skin redness (erythema\*), palpable heat, swelling (edema\*), fever or fluid retention as these could be signs of infection\*.

Proceed in the same way should you notice a change in colour of the drained effusion.

Redness and slight pain are to be expected shortly after implantation, but these should not persist or get worse.

## Returns / Refunds

Only originally packed (unopened, unmarked or damaged) cartons containing Drainage Kits will be accepted for return and refund.

## Glossary

<b>Ascites</b>	Effusion or accumulation of fluid in the abdomen (stomach)
<b>DEHP</b>	Plasticizers in plastics
<b>Protein loss</b>	Reduction of the amount of protein in the blood (protein depletion)
<b>Electrolyte imbalance</b>	Disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or cells in the body
<b>Erythema</b>	Skin redness
<b>Heimlich valve</b>	Protection against backflow for increased patient safety
<b>Hepatorenal syndrome</b>	Decrease in renal function
<b>Hypotension</b>	Low blood pressure
<b>Infection</b>	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
<b>Incision</b>	Skin incision during the operation e.g. for the implantation of the catheter
<b>Clot</b>	Blood clot
<b>Circulatory collapse</b>	Temporary loss of consciousness
<b>Edema</b>	Swelling due to excessive fluid accumulation
<b>Peritoneal cavity</b>	Abdominal cavity
<b>Peritonitis</b>	Infection of the peritoneum
<b>Phthalate</b>	Plasticizer (e.g. DEHP)
<b>Phthalate exposure</b>	Come into contact with phthalates
<b>Pleural effusion</b>	Effusion or fluid accumulation in the pleural cavity

<b>Pneumothorax</b>	Air in the space between the lungs and the chest wall
<b>Reexpansion pulmonary edema</b>	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
<b>Septa</b>	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)
<b>Subcutaneous tissue</b>	Tissue under the skin

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PleurX™ is a trademark and /or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and /or registered trademark of ewimed.
**Sweden** | fenik AB | Ekbacksvägen 28 | 168 69 Bromma | Tel.: +46 825 1169 | [info@fenik.se](mailto:info@fenik.se)