ewimed

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## CONTENT

- 2 Explanation of Symbols
- 6 Product Description
- 15 ewimed gravity reservoir – 2000 ml
- 15 Warnings
  - general
  - in relation to your catheter
- 17 Connecting the reservoir
- 18 Final steps
- 18 Returns/refunds
- 19 Glossary

EN

**Read the entire instruction manual carefully before using the ewimed gravity reservoir – 2000 ml and draining the effusion with it.**

## YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

DE: +49 7471 73972-0  
info@ewimed.com

AT: +43 2231 22500  
info@ewimed.com

EN: +49 7471 73972-0  
info@ewimed.com

SE: +46 8 25 11 69  
info.se@ewimed.com

CH: +41 52 577 02 55  
info.ch@ewimed.com

DA: +45 31 43 50 60  
info.dk@ewimed.com

For further detailed information please visit our website [www.ewimed.com](http://www.ewimed.com)

## EWIMED GRAVITY RESERVOIR – 2000 ML

The ewimed gravity reservoir – 2000ml is designed to be connected to a BD PleurX™ / PeritX™ or ewimed drainova® catheter system. In this application it is used to collect pleural effusion and ascites.

The ewimed gravity reservoir – 2000ml will hereinafter be referred to as „reservoir“.

**Minimum order quantity 1 PU = 10 pieces, sterile**

EN

## WARNINGS

general

### Do not use this product if:

- the sterile packaging is damaged
- damaged
- the expiry date has elapsed

### The drainage consumables are intended for single use only!

The drainova® Reservoirs are delivered in sterile packaging which have undergone quality controls. The drainova® Reservoir can not be re-sterilised.

The product is intended for single use only. Re-use could contribute to a cross-contamination. ewimed GmbH accepts no liability for re-used, re-sterilised products or products taken from damaged packaging.

### Drainage of the chest cavity

Only drain the amount of fluid prescribed by your doctor.

ewimed does not recommend draining more than 1000ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch

clamp of the connection tube in order to pause the flow.

Possible complications relating to repeated drainage of pleural effusions\* include pneumothorax\*, re-expansion pulmonary oedema\*, hypotension\*, circulatory collapse\*, septa\* in the chest cavity or infection\*.

### Drainage of the peritoneal cavity

Only drain the amount of fluid prescribed by your doctor.


ewimed does not recommend draining more than 2000ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in order to pause the flow.

Possible complications relating to repeated drainage of ascites\* include hypotension\*, circulatory collapse\*, electrolyte imbalance\*, loss of protein\*, hepatorenal syndrome\*, leakage of ascites from incision sites, peritonitis\*, septa\* in the peritoneal cavity\* and infection.

## 16 | Warnings

### Position of the reservoir during drainage

As the effusion is drained by gravity, make sure that the reservoir is always situated below the catheter exit site during drainage.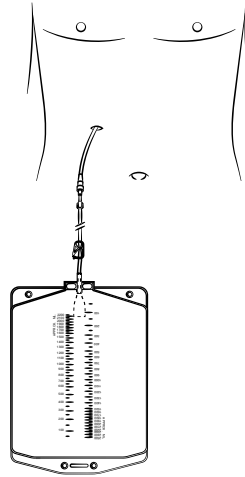

EN

### Cleaning/Disinfection

If effusion is leaking out and comes in contact with skin, clean the skin with soap and water. For surfaces, use suitable cleaning products or surface disinfectant.

Please observe the reservoir change intervals according to the hygiene regulations of your institution.

## WARNINGS

in relation to your catheter

Never cut through the catheter or cut the safety valve off.

Ensure that the catheter does not come into contact with any sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i.e. is no longer attached), proceed as follows:


- firmly pinch the catheter closed with your fingers.
- using your other hand, hold the emergency slide clamp open so that the catheter can be fed through the open end of the clamp.
- push the catheter completely through the narrow end of the clamp.

### Use only original accessories.

Only original accessories may be connected to the catheter. Do not insert anything other than the specially-designed access tip into the safety valve, otherwise the safety valve may become damaged. Improper handling can cause air to enter the body or fluid to permanently leak out.

### Keep the safety valve on the catheter clean and the access tip on the connection tube sterile.

Ensure that they do not come into contact with non-sterile objects to avoid contamination. The safety valve must be disinfected before each drainage.

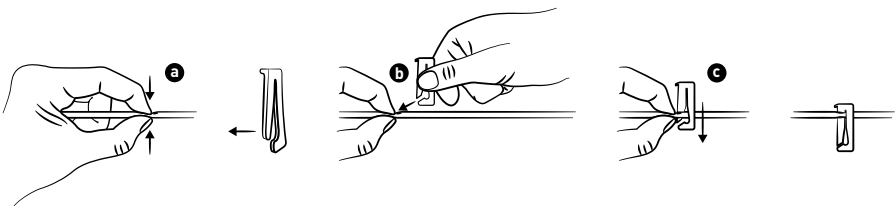**While draining, make sure that the safety valve and the access tip are securely connected.**

**Take care not to tug or pull on the connected tube.**

Take particular care to ensure that the access tip is inserted in the safety valve in a sterile manner and make sure that the connection is secure. It can lead to contamination if the parts become dis-

connected by mistake. In case this occurs, disinfect the safety valve with a new alcohol wipe and dispose of the disconnected drainage material. Use a new sterile drainova® Reservoir to continue with the drainage. The drainova® clickFix can be used to prevent unintentional disconnection.

## CONNECTING THE RESERVOIR

1. Remove the drainova® Silicone Cap or the drainova® Protective Cap:


- **Using the drainova® Silicone Cap:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® Silicone Cap. Dispose of the cap.


- **Using the PleurX™ / PeritX™ Protective Cap:**

Firmly hold the safety valve of the catheter, rotate the PleurX™ / PeritX™ Protective Cap anticlockwise and carefully remove. Dispose of the cap.

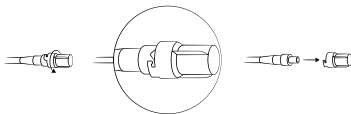
2. Hold the safety valve in one hand and clean the area around the safety valve-opening with an alcohol wipe or a suitable alcohol-based disinfectant.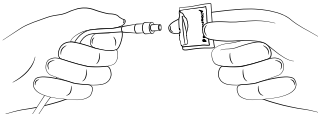

3. Open the packaging and take the reservoir out. The reservoir in the packaging is sterile. It is important that the access tip on the connection tube remains sterile.

4. Completely close the pinch clamp on the connection tube.

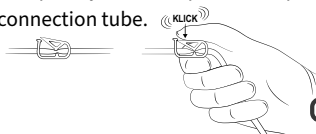
5. Continue to hold the access tip at the base and carefully remove the protective film from the access tip.


- 5.1 Insert the access tip in the safety valve.

Ensure that the safety valve and the access tip are securely connected during drainage. A click can be heard and felt when connected.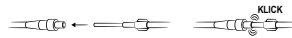


6. The reservoir should always be kept underneath the catheter exit site, as drainage is aided by gravity.
7. Open the pinch clamp on the tubing so that drainage can begin.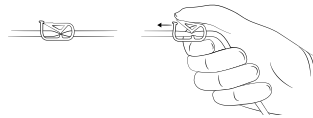

**Please note:** The flow can be slowed or interrupted when:

- The fluid has almost completely drained
- The fluid has a high viscosity
- The safety valve is blocked due to a clot\* or protein thread

8. When the drainage volume recommended by the doctor has been reached, stop drainage by closing the pinch clamp.

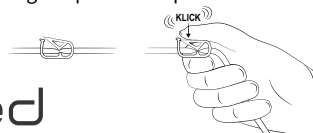## FINAL STEPS

EN

1. Open the packaging of the drainova® Silicone Cap or the PleurX™/PeritX™ Protective Cap. But to begin with, leave the cap in the packaging.
2. Take the tubing in one hand and the safety valve in the other hand. Pull the access tip in a firm, smooth motion out of the safety valve. Place the tubing down.

3. Clean the safety valve for approx. 30 seconds with an alcohol wipe or other suitable alcohol-based disinfectant and then dispose of the used wipe.

4. Attach either the drainova® Silicone Cap or PleurX™/PeritX™ Protective Cap:


- **Using the drainova® Silicone Cap:**

Securely attach the new sterile drainova® Silicone Cap onto the safety valve of the catheter. The cap will stay in place without the need for additional support.

- **Using the PleurX™/PeritX™ Protective Cap:**

Attach the new PleurX™/PeritX™ Protective Cap onto the safety valve and turn clockwise until it clicks into place.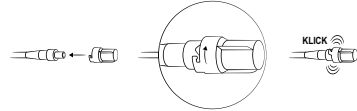

## RETURNS / REFUNDS

Only originally packaged (unopened, non-damaged, non-labelled) boxes of reservoirs will be accepted for return and refund.

## GLOSSARY

<b>Ascites</b>	Effusion or accumulation of fluid in the abdomen (stomach)
<b>Circulatory collapse</b>	Temporary loss of consciousness
<b>DEHP</b>	Plasticizers in plastics
<b>Edema</b>	Swelling due to excessive fluid accumulation
<b>Electrolyte imbalance</b>	Disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or cells in the body
<b>Heimlich valve</b>	Protection against backflow for increased patient safety
<b>Hepatorenal syndrome</b>	Decrease in renal function
<b>Hypotension</b>	Low blood pressure
<b>Incision</b>	Skin incision during the operation e.g. for the implantation of the catheter
<b>Infection</b>	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
<b>Peritoneal cavity</b>	Abdominal cavity
<b>Peritonitis</b>	Infection of the peritoneum
<b>Phthalate</b>	Softening agents (e.g. DEHP)
<b>Pleural effusion</b>	Effusion or fluid accumulation in the pleural cavity
<b>Pneumothorax</b>	Air in the space between the lungs and the chest wall
<b>Protein loss</b>	Reduction of the amount of protein in the blood (protein depletion)
<b>Reexpansion pulmonary edema</b>	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
<b>Septa</b>	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)

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PleurX™ / PeritX™ is a trademark and / or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and / or registered trademark of ewimed.
j. söllner AG
**Germany** | ewimed GmbH | Dorfstrasse 24 | 72379 Hechingen | Tel.: +49 7471 73972-0 | info@ewimed.com
**Sweden** | ewimed Sweden AB | Ekbacksvägen 28 | 168 69 Bromma | Tel.: +46 8 25 11 69 | info.se@ewimed.com