ewimed

DE ewimed Drainage-Set, 2000 ml

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EN ewimed drainage kit, 2000 ml

Instructions for use

FR ewimed kit de drainage, 2000 ml

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istruzioni d'uso

SV ewimed dränageset, 2000 ml

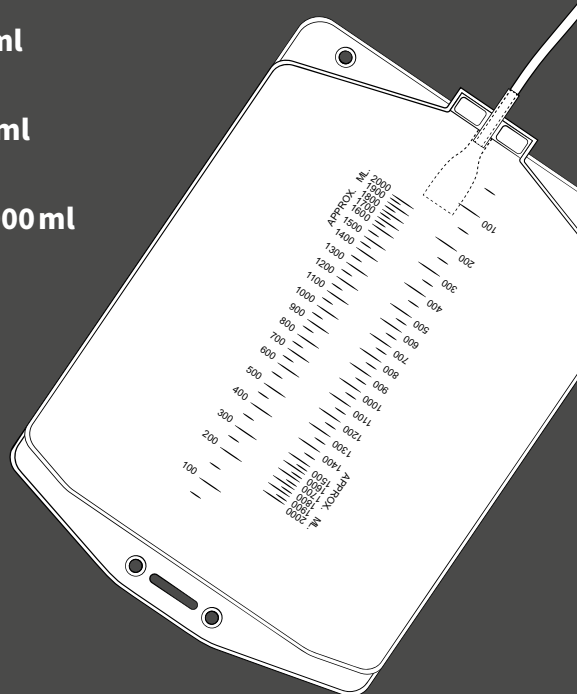
Bruksanvisning

DA ewimed drænagesæt 2000 ml

Brugsanvisning

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REF 50-7505

DE ERKLÄRUNG DER SYMBOLE
EN EXPLANATION OF SYMBOLS
FR EXPLICATION DES SYMBOLES
IT SPIEGAZIONI DEI SIMBOLI

SV FÖRKLARING AV SYMBOLERNA
DA SYMBOLFORKLARING
HU SZIMBÓLUMOK MAGYARÁZATA

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CONTENT

2	Explanation of symbols	24	Preparation
6	Drainage Log Chart	25	Drainage procedure
7	Product description	26	Final steps
22	The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ or PeritX™ Catheter	27	Applying a new dressing
22	The ewimed Drainage Kit	28	Emptying the reservoir
23	Warnings	29	Frequently Asked Questions
	general	30	Returns / Refunds
	in combination with the catheter	30	Glossary

EN

You should have already been instructed in the correct use of the components in this set either by ewimed personnel training staff, district nursing staff or a doctor. Please contact your doctor or district nurse if you are experiencing difficulties or have questions relating to effusion drainage.

Please carefully read the instructions before using the ewimed Drainage Kit.

All explanations of terms (*) can be found in the glossary on page 30.

YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

DE: +49 7471 73972-0
info@ewimed.com

AT: +43 2231 22500
info@ewimed.com

HU: +36 1 998 8295
info@ewimed.hu

EN: +49 7471 73972-0
info@ewimed.com

SE: +46 8 25 11 69
info.se@ewimed.com

CH: +41 52 577 02 55
info.ch@ewimed.com

DA: +45 31 43 50 60
info.dk@ewimed.com

For further detailed information please visit our website www.ewimed.com

THE DRAINOVA® CATHETER, DRAINOVA® ARGENTIC CATHETER, PLEURX™ OR PERITX™ CATHETER

The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ or PeritX™ Catheter serve to drain fluid from the chest cavity (pleural effusion*) or the peritoneal cavity (ascites*). Your doctor has implanted this catheter so that you can independently drain the pleural effusion* or the ascites* at home according to your needs - rendering repeated visits to hospital for puncturing unnecessary. The implanted part of the catheter is perforated with drainage holes, which allow the effusion to drain out. During implantation, the polyester cuff is placed in a subcutaneous tunnel to prevent infections*. The safety valve on the visible part of

the catheter is generally closed, preventing both leakage of effusion and air from entering.

A Valve Cap or a drainova® Silicone Cap, sterile is attached to the safety valve to protect it from contamination or damage when not draining. To drain effusion fluids use the drainova® Drainage Set or the ewimed Drainage Set. The key-lock-principle ensures the optimal connection of the drainage sets with the safety valve.

The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ Catheter or PeritX™ Catheter will hereinafter be referred to as ‚catheter‘.

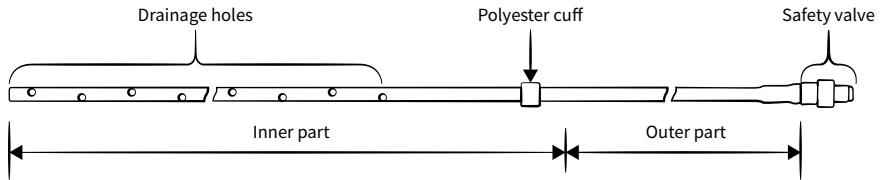
Figure: Illustration of a drainova® catheter

THE EWIMED DRAINAGE KIT

The ewimed Drainage Kit contains sterile materials that you will need for the safe and easy drainage of effusion with your catheter.

The gravity reservoir contained in the ewimed Drainage Kit will hereinafter be referred to as the ‚reservoir‘.

All products are CE labelled and have been sterilized with ethylene oxide gas (see individual packaging).

For the drainage procedure, we additionally recommend:

- Alcohol-based skin disinfectant spray
- Hand disinfectant
- Non-sterile disposable gloves

WARNINGS

general

Do not use this product if:

- The sterile packaging is damaged
- The product is damaged
- The expiry date has expired

The ewimed Drainage kit is only suitable for use for the intended purpose.

The consumables are intended for single use only!

The ewimed Drainage Kit is delivered in a sterile condition, which is ensured by appropriate quality controls. The Drainage Kit must not be re-sterilised.

The product is for single-use only. Re-use can lead to cross-contamination.

ewimed GmbH accepts no liability for products that have been re-used, re-sterilised or removed from damaged packaging.

Drainage from the chest cavity

Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 1000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage. Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of pleural effusion* include pneumothorax*, re-expansion pulmonary edema*, hypotension*, circulatory collapse*, septa* in the chest cavity or infection*.

Peritoneal drainage

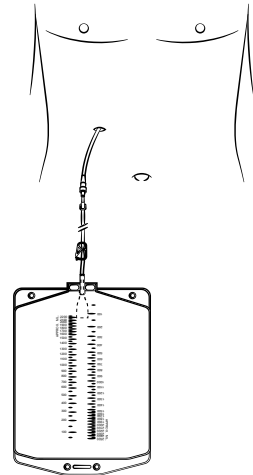
Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 2000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage. Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of ascites* include hypotension*, circulatory collapse*, electrolyte imbalance*, protein loss*, hepatorenal syndrome*, leakage of ascites* through the incisions*, peritonitis*, septa* in the peritoneal cavity* and infection*.

Position of the reservoir during drainage

As the effusion is drained by gravity, make sure that the reservoir is always situated below the catheter exit site during drainage.

Cleaning / Disinfection

If effusion is leaking out and comes in contact with skin, clean the skin with soap and water. For surfaces, use suitable cleaning products or surface disinfectant. Do not use Octenisept or disinfectants containing iodine for cleaning / disinfecting the catheter or safety valve.

The alcohol wipes are highly flammable.

Keep the wipes away from naked flames.

WARNINGS

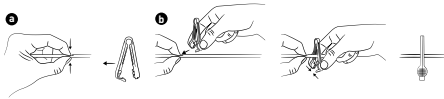
in combination with the catheter

Never cut through the catheter or cut the safety valve off.

Avoid contact of the catheter with sharp objects

If the functionality of the safety valve can no longer be guaranteed or if the safety valve has been accidentally cut off or is no longer present, proceed as follows:

- Firmly squeeze the catheter between your fingers.
- Hold the emergency clamp with the other hand and close the clamp with the inserted catheter until a click is heard.
- Immediately contact the clinic where the implantation took place.

Use only original accessories.

Only original accessories may be connected to the catheter. Do not insert anything other than the specially-designed access tip into the safety valve, otherwise the safety valve may become damaged. Improper handling can cause air to enter the body or fluid to permanently leak out.

Keep the safety valve on the catheter clean and the access tip on the connection tube sterile.

Ensure that they do not come into contact with non-sterile objects to avoid contamination. The

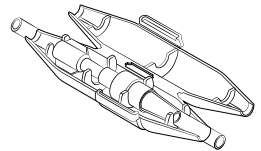
safety valve must be disinfected before each drainage.

When draining, ensure that the access tip and the safety valve are securely connected. Make sure that the connection tube is not tugged on or pulled.

Connect the access tip to the safety valve in a sterile manner and make sure that they are firmly connected. Accidental disconnection may result in contamination. If this happens, disinfect the safety valve using a new alcohol wipe and discard the disconnected drainage material. A new sterile Drainage Kit must be used to continue drainage. Please observe the local, regional and national waste disposal regulations.

Please note: If required (for additional protection against accidental disconnection) the connection can be secured with the separately available drainova® clickFix (catalogue number P1100 / P1100S) .

The drainova® clickFix can be additionally prescribed. It is not contained in the Drainage Kit.

PREPARATION

The effusion should be drained with an ewimed Drainage Kit according to your doctor's instructions. If you have not received a medical

recommendation for the drainage interval or drainage volume, you will receive a recommendation during training.

1. Prepare a clean, clear work space on a table or other work surface.

Please note: When disposing of used material, please observe the local, state and national regulations. If necessary, place a waste container ready.

slowly pull away from the skin. Make sure not to accidentally pull on the catheter.

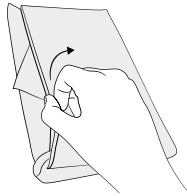
Important: Inform your doctor should you notice swelling, signs of inflammation or an accumulation of fluid at the catheter exit site or incision site.

2. Disinfect your hands. We also recommend washing your hands thoroughly beforehand.
3. Put on the non-sterile disposable gloves.
4. Remove the dressing covering the catheter. Carefully lift off one corner of the dressing and
5. Take off the gloves and disinfect your hands again.
6. Open the sterile packaging of the ewimed Drainage Kit.
7. Open the sterile packaging of the dressing set by pulling the two sides of the bag apart at the top.

DRAINAGE PROCEDURE

1. Place the dressing set on the work surface with the flap facing upwards. Carefully unfold the blue drape. Leave the items contained on the drape.

The items in the dressing set are sterile and must not come into contact with non-sterile objects.

2. Holding the gloves at the opening, put them on as shown. Both gloves are suitable for both hands. Make sure that no non-sterile objects, such as skin or clothing, are touched with the gloves.

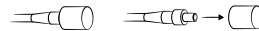
- 2.1 Tear open the alcohol wipe sachets on one side. Do not remove the alcohol wipes from the sachets. Open each sachet in preparation to place them onto the safety

- valve later on and place the sachets back on the edge of the blue drape.

3. Remove the drainova® silicone cap, sterile or the PleurX™ / PeritX™ Valve Cap:


• **When using the drainova® silicone cap, sterile:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® Silicone Cap, sterile. Dispose of the cap.


• **When using the PleurX™ / PeritX™ Valve Cap:**

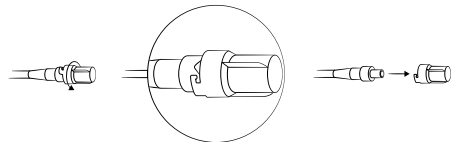Firmly hold the safety valve of the catheter, turn the Valve Cap anticlockwise and carefully remove. Dispose of the cap.

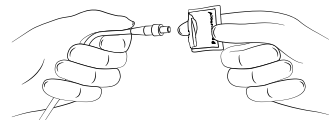
4. Insert the safety valve into one of the already opened alcohol wipe sachets. Gently rotate to clean the safety valve. Leave the safety valve in the sachet while carrying out steps 5 + 6.

Important: Do not use octenisept® or disinfectants containing iodine.

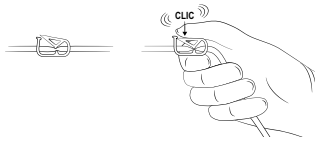
- Open the packaging and take the reservoir out. The reservoir in the packaging is sterile. It is important that the access tip on the connection tube remains sterile. Place the reservoir next to the blue drape and hold the access tip at the base in one hand. Completely close the pinch clamp on the connection tube with your other hand.


- Whilst continuing to hold the access tip at the base, carefully pull the protective cover off the access tip and discard.

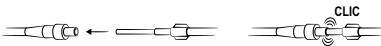- Remove the alcohol wipe sachet from the safety valve and insert the access tip into the safety valve. Make sure that the safety valve and the access tip are firmly connected. You can hear and feel it click into place.

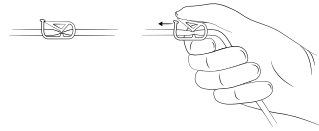- The reservoir should always be positioned below the catheter exit site, as drainage is aided by gravity.
- Open the pinch clamp on the connection tube so that drainage can begin.


Please note: Flow can be slowed or interrupted if:

- The effusion is almost completely drained
- The effusion has a high viscosity
- The safety valve is blocked by a coagel* or protein thread


Additional information on how to proceed in such a case can be found in the 'Frequently Asked Questions' section starting on page 29.

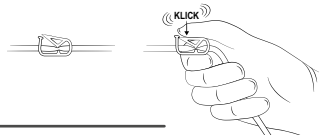- When the drainage volume recommended by the doctor has been reached, stop drainage by closing the pinch clamp. Make note of the drainage volume in the 'Drainage Log Chart' on page 6.

FINAL STEPS

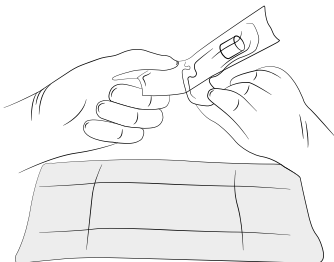- Open the drainova® Silicone Cap, sterile packaging and let the cap drop onto the blue drape.

- Take the connection tube in one hand and the safety valve in the other hand. Pull the access tip out of the safety valve in one firm, smooth motion. Set down the connection tube.

- Clean the safety valve with an alcohol wipe for about 30 seconds and then dispose of the wipe.

Important: Do not use octenisept® or disinfectants containing iodine.

- Attach the new sterile drainova® silicone cap firmly onto the safety valve of the catheter.

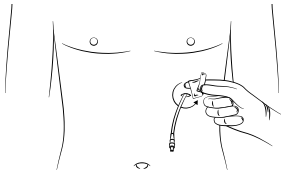
The cap will remain in place without the need for additional fixation.


APPLYING A NEW DRESSING

EN

- Clean the area around the catheter exit site with the alcohol-based skin disinfection spray. Ensure an exposure time of 30 seconds. Please also ensure that afterwards the catheter exit site and the skin around the catheter are dry. Only then apply the new dressing.

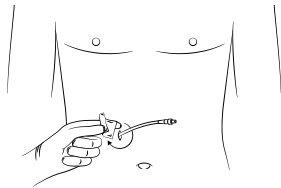
Disinfecting the pleural* catheter exit site


Disinfecting the ascites* catheter exit site

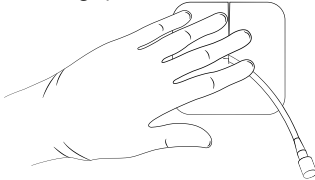
- Place the foam pad around the catheter with the slit facing upwards.

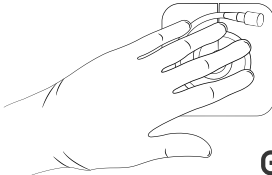- Loop the catheter and position it on the foam pad.

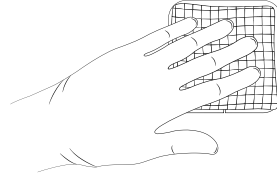- Cover the catheter with the gauze pads.


- Take off both gloves.

- The film dressing has a backing sheet on both sides.


- The printed backing sheet consists of a centre section and 2 side edge strips. The self-adhesive side of the film dressing is located underneath.

- The unprinted backing sheet consists of a centre section and an outer frame.

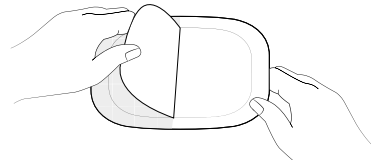- First remove the centre part of the unprinted backing sheet.

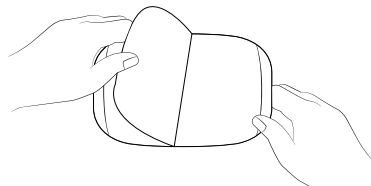
- Then remove the centre part of the printed backing sheet to expose the adhesive side. You can do this by slightly bending the side edge strips.


28 | Emptying the reservoir

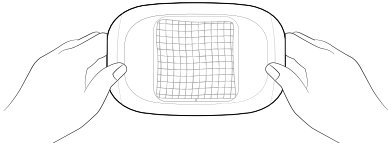9. Hold the backing sheet on the edge strips. You now have a window on the unprinted side to help you position the film dressing. Stick the film dressing centrally over the catheter.

EN

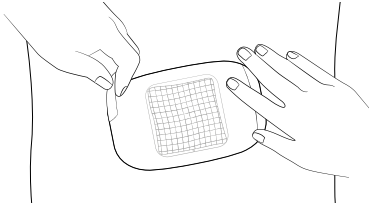10. Next, remove the frame of the unprinted backing sheet and smooth the film dressing from the inside outwards (there is a perforated area on the edge to start peeling the frame off in a circle).

11. As a last step, remove the two edge strips by holding the film dressing (close to the edge strips) in place with one hand and then tearing the edge strips off in one piece with the other hand. The dressing is perforated at this point and tears off together with the backing sheet. Finally smooth out the film dressing again.

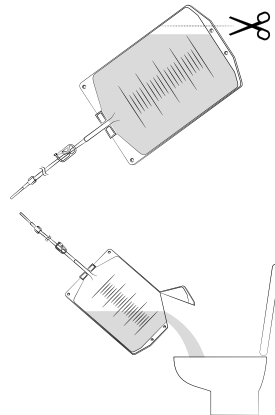
Please note: Dispose of the used dressing according to the local regulations.

EMPTYING THE RESERVOIR

1. Cut off a corner of the reservoir using scissors and empty the contents into the toilet.

Please note: Used products and effusion fluid may pose biological risks. Therefore, please observe local and national regulations when disposing of used products and effusion.

2. Place the reservoir in a plastic bag, seal it firmly and dispose of the bag in accordance with local regulations.

FREQUENTLY ASKED QUESTIONS

When can the sutures (stiches) be removed?

The incision sutures can typically be removed after 7-10 days. The suture holding the catheter in place however, should not be removed before 30 days have elapsed.

How frequently should I drain?

Follow your doctor's instructions regarding the drainage quantity.

The effusion fluid should be drained using a Drainage Kit every one or two days or as instructed by the doctor. Consult your doctor if you want to drain more or less frequently. ewimed recommends draining a maximum of 1000 ml of effusion from the chest cavity. ewimed recommends draining a maximum of 2000 ml of effusion from the abdominal cavity.

What should I do if I still feel short of breath after draining?

If you still have difficulty breathing after successfully draining effusion fluid from the chest or abdominal cavity, inform your doctor.

What should be done if the effusion volume decreases or no effusion can be drained at all?

There can be two reasons for this:

1. If the amount of fluid during the last drainage deviates greatly from the usual amount or if no effusion at all can be drained, the catheter may be blocked or a protein thread may be obstructing the access tip, preventing drainage. Try to remove the blockage by squeezing and using stroking movements along the catheter.

If you are unable to remove the blockage, consult your doctor or contact the company central office (see contact details on page 21).

2. If the drainage amounts continuously decrease and you have no complaints or weight gain, it is possible that no effusion has accumulated. Discuss next steps with your doctor.

Can I shower or take a bath with the catheter?

If the catheter is completely and correctly covered with the self-adhesive dressing, you can shower. If however, the gauze pads get wet when showering, remove the dressing, clean and dry the area and apply a new dressing. The catheter should not be immersed under water in a bathtub or swimming pool. Avoid direct contact of the safety valve with bath or shower water.

Please note: We recommend showering before drainage / changing the dressing, so that the dressing can be promptly changed if it gets wet.

What should I do if I accidentally pull out the catheter?

The catheter should be adhered to the subcutaneous* skin tissue by the polyester cuff. If the catheter is accidentally pulled out and the cuff is visible, cover the catheter exit site with a sterile dressing and immediately contact the clinic where the implantation took place.

How can I tell if I have an infection*?

Immediately inform a doctor if you experience pain, skin redness (erythema*), palpable heat, swelling (edema*), fever or fluid retention as these could be signs of infection*.

Proceed in the same way should you notice a change in colour of the drained effusion. Redness and slight pain are to be expected shortly after implantation, but these should not persist or get worse.

RETURNS / REFUNDS

Only originally packed (unopened, unmarked or damaged) cartons containing Drainage Kits will be accepted for return and refund.

EN

GLOSSARY

Ascites	Effusion or accumulation of fluid in the abdomen (stomach)
Circulatory collapse	Temporary loss of consciousness
Clot	Blood clot
DEHP	Plasticizers in plastics
Edema	Swelling due to excessive fluid accumulation
Electrolyte imbalance	Disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or cells in the body
Erythema	Skin redness
Heimlich valve	Backflow prevention for increased patient safety
Hepatorenal syndrome	Decrease in renal function
Hypotension	Low blood pressure
Incision	Skin incision during the operation e.g. for the implantation of the catheter
Infection	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
Peritoneal cavity	Abdominal cavity
Peritonitis	Infection of the peritoneum
Phthalate	Plasticizer (e.g. DEHP)
Pleural effusion	Effusion or fluid accumulation in the pleural cavity
Pleura	Thin, serous skin in the thoracic cavity (pleura)

Protein loss	Reduction of the amount of protein in the blood (protein depletion)
Reexpansion pulmonary edema	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
Septa	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)
Subcutaneous tissue	Tissue under the skin

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed GmbH.
octenisept® is a trademark of Schülke & Mayr GmbH.
ewimed Switzerland AG
Germany | ewimed GmbH | Dorfstrasse 24 | 72379 Hechingen | Tel.: +49 7471 73972-0 | info@ewimed.com
Sweden | ewimed Sweden AB | Ekbacksvägen 28 | 168 69 Bromma | Tel.: +46 8 25 11 69 | info.se@ewimed.com