ewimed

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### CONTENT

- 2 Explanation of symbols
- 8 Product description
- 17 ewimed gravity reservoir – 2000 ml
- 17 Warnings
  - general
  - in combination with the catheter
- 19 Connecting the reservoir
- 20 Final steps
- 20 Returns/refunds
- 21 Glossary

EN

You should have already been instructed in the correct use of the components in this set either by ewimed training personnel, district nursing staff or a doctor. Please contact your doctor or district nurse if you are experiencing difficulties or have questions relating to effusion drainage.

**Read the entire instruction manual carefully before using the ewimed gravity reservoir – 2000 ml and draining the effusion with it.**

### YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

**DE:** +49 7471 73972-0  
info@ewimed.com

**SE:** +46 8 25 11 69  
info.se@ewimed.com

**BE:** +49 7471 73972-0  
info@ewimed.com

**EN:** +49 7471 73972-0  
info@ewimed.com

**DA:** +45 31 43 50 60  
info.dk@ewimed.com

**NL:** +49 7471 73972-0  
info@ewimed.com

**CH:** +41 31 958 66 66  
info.ch@ewimed.com

**NO:** +46 8 25 11 69  
info.no@ewimed.com

**LU:** +49 7471 73972-0  
info@ewimed.com

**AT:** +43 2231 22500  
info@ewimed.com

**HU:** +36 1 998 8295  
info@ewimed.hu

For further detailed information please visit our website [www.ewimed.com](http://www.ewimed.com)

## EWIMED GRAVITY RESERVOIR – 2000 ML

The ewimed gravity reservoir – 2000 ml is designed to be connected to a BD PleurX™ / PeritX™ or ewimed drainova® catheter system. In this application it is used to collect pleural effusion and ascites.

The ewimed gravity reservoir – 2000 ml will hereinafter be referred to as „reservoir“.

Minimum order quantity 1 PU = 10 pieces, sterile

EN

## WARNINGS

general

**The reservoir is only suitable for use for the intended purpose.**

**Do not use this product if:**

- the sterile packaging is damaged
- the product is damaged
- the use-by date has elapsed

**The consumables are intended for single use only!**

The drainova® reservoirs are delivered in sterile condition, which have undergone quality controls. The drainova® reservoir must not be re-sterilised.

The product is intended for single use only. Re-use could contribute to a cross-contamination. ewimed GmbH accepts no liability for products that have been re-used, re-sterilised or removed from damaged packaging.

**Drainage of the chest cavity**

Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 1000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your

doctor's instruction. You may feel an unpleasant pulling sensation during drainage.

Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of pleural effusion\* include pneumothorax\*, re-expansion pulmonary edema\*, hypotension\*, circulatory collapse\*, septa\* in the chest cavity or infection\*.

**Peritoneal drainage**

Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 2000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage.

Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of ascites\* include hypotension\*, circulatory collapse\*, electrolyte imbalance\*, protein loss\*, hepatorenal syndrome\*, leakage of ascites\* through the incisions\*, peritonitis\*, septa\* in the peritoneal cavity\* and infection\*.

## 18 | Warnings

### Position of the reservoir during drainage


As the effusion is drained by gravity, make sure that the reservoir is always situated below the catheter exit site during drainage.

### Cleaning / Disinfection

If effusion is leaking out and comes in contact with skin, clean the skin with soap and water. For surfaces, use suitable cleaning products or surface disinfectant.

Do not use octenisept® or disinfectants containing iodine for cleaning / disinfecting the catheter or safety valve.

Please observe the reservoir change intervals according to the hygiene regulations of your institution.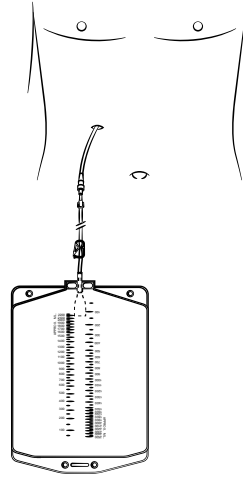

EN

## WARNINGS


in combination with the catheter

### Never cut through the catheter or cut the safety valve off.

Avoid contact of the catheter with sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i.e. is no longer attached), proceed as follows:

- firmly pinch the catheter closed with your fingers.
- using your other hand, hold the emergency slide clamp open so that the catheter can be fed through the open end of the clamp.
- push the catheter completely through the narrow end of the clamp.
- Immediately inform your doctor.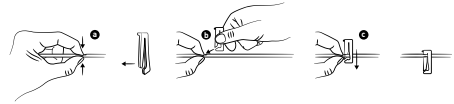

### Use only original accessories.

Only original accessories may be connected to the catheter. Do not insert anything other than the specially-designed access tip into the safety valve, otherwise the safety valve may become damaged. Improper handling can cause air to enter the body or fluid to permanently leak out.

### Keep the safety valve on the catheter clean and the access tip on the connection tube sterile.

Ensure that they do not come into contact with non-sterile objects to avoid contamination. The safety valve must be disinfected before each drainage.

**When draining, ensure that the access tip and the safety valve are securely connected. Make sure that the connection tube is not tugged on or pulled.**

Connect the access tip to the safety valve in a sterile manner and make sure that they are firmly connected. Accidental disconnection may result in contamination. If this happens, disinfect the safety valve using a new alcohol swab and discard the disconnected drainage material.


A new sterile reservoir must be used to continue drainage.

Please observe the local waste disposal regulations.

If required (for additional protection against accidental disconnection) the connection can be secured with the separately available drainova® clickFix (Art. No. P1100).

The drainova® clickFix can be additionally prescribed. It is not contained in the drainage kit.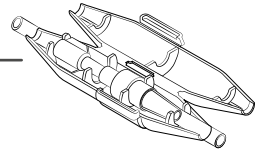


## CONNECTING THE RESERVOIR


- Loosen the drainova® silicone cap, sterile or the drainova® valve cap:
  - **Using the drainova® silicone cap, sterile:**  
Firmly hold the safety valve of the catheter and carefully remove the drainova® silicone cap, sterile. Dispose of the cap.


- **Using the PleurX™ / PeritX™ valve cap:**  
Firmly hold the safety valve of the catheter, rotate the PleurX™ / PeritX™ valve cap anticlockwise and carefully remove. Dispose of the cap.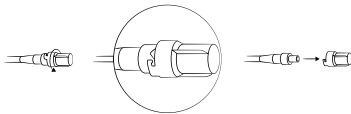


- Hold the safety valve in one hand and clean the area around the safety valve-opening with an alcohol wipe or a suitable alcohol-based disinfectant.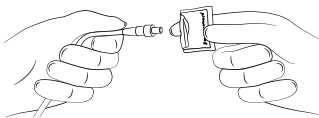


- Open the packaging and take the reservoir out. The reservoir in the packaging is sterile. It is important that the access tip on the connection tube remains sterile.
- Completely close the pinch clamp on the connection tube.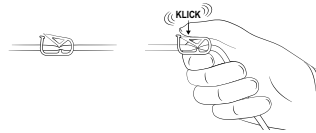

- Continue to hold the access tip at the base and carefully remove the protective film from the access tip.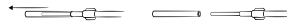


- 5.1 Insert the access tip in the safety valve.  
Ensure that the safety valve and the access tip are securely connected during drainage. A click can be heard and felt when connected.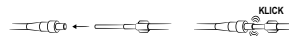


- The reservoir should always be kept underneath the catheter exit site, as drainage is aided by gravity.

## 20 | Final steps

7. Open the pinch clamp on the tubing so that drainage can begin.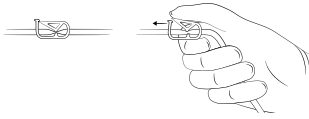

8. When the drainage volume recommended by the doctor has been reached, stop drainage by closing the pinch clamp.

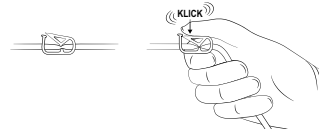EN

**Please note:** The flow can be slowed or interrupted when:

- The fluid has almost completely drained
- The fluid has a high viscosity
- The safety valve is blocked due to a clot\* or protein thread

## FINAL STEPS

1. Open the packaging of the drainova® silicone cap, sterile or the PleurX™/PeritX™ valve cap. But to begin with, leave the cap in the packaging.
2. Take the tubing in one hand and the safety valve in the other hand. Pull the access tip in a firm, smooth motion out of the safety valve. Place the tubing down.


3. Clean the safety valve with an alcohol wipe or other suitable alcohol-based disinfectant and then dispose of the used wipe.

4. Attach either the drainova® silicone cap, sterile or PleurX™/PeritX™ valve cap:
  - **Using the drainova® silicone cap, sterile:** Securely attach the new sterile drainova® silicone cap, sterile onto the safety valve of the catheter. The cap will stay in place without the need for additional support.

- **Using the PleurX™/PeritX™ valve cap:** Attach the new PleurX™/PeritX™ valve cap onto the safety valve and turn clockwise until it clicks into place.

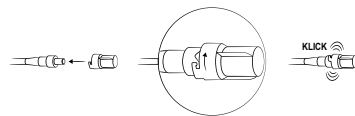## RETURNS / REFUNDS

Only originally packaged (unopened, non-damaged, non-labelled) boxes of reservoirs will be accepted for return and refund.

## GLOSSARY

<b>Ascites</b>	Effusion or accumulation of fluid in the abdomen (stomach)
<b>Circulatory collapse</b>	Temporary loss of consciousness
<b>DEHP</b>	Plasticizers in plastics
<b>Electrolyte imbalance</b>	Disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or cells in the body
<b>Heimlich valve</b>	Protection against backflow for increased patient safety
<b>Hepatorenal syndrome</b>	Decrease in renal function
<b>Hypotension</b>	Low blood pressure
<b>Incision</b>	Skin incision during the operation e.g. for the implantation of the catheter
<b>Infection</b>	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
<b>Peritoneal cavity</b>	Abdominal cavity
<b>Peritonitis</b>	Inflammation of the peritoneum (abdominal cavity)
<b>Phthalate</b>	Plasticizer (e.g. DEHP)
<b>Pleural effusion</b>	Effusion or fluid accumulation in the pleural cavity
<b>Protein loss</b>	Reduction of the amount of protein in the blood (protein depletion)
<b>Reexpansion pulmonary edema</b>	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
<b>Septa</b>	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)
<b>Reexpansion pulmonary edema</b>	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
<b>Septa</b>	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed GmbH.
octenisept® is a trademark of Schülke & Mayr GmbH.
j. söllner AG
Cosanum AG
ewimed Switzerland AG