- DE **drainova® Spülschlauch**
Gebrauchsanweisung
- EN **drainova® Rinsing Tube**
Instructions for use
- FR **drainova® ligne de rinçage**
Instructions
- IT **drainova® tubo di irrigazione**
Istruzioni per l'uso
- SV **drainova® spolslang**
Bruksanvisning
- DA **drainova® skylleslange**
Brugsanvisning

DE ERKLÄRUNG DER SYMBOLE
EN EXPLANATION OF SYMBOLS
FR EXPLICATION DES SYMBOLES
IT SPIEGAZIONI DEI SIMBOLI
SV FÖRKLARING AV SYMBOLERNA
DA SYMBOLFORKLARING

de Bitte beachten Sie, dass nicht alle dargestellten Symbole auf Ihr Produkt zutreffen. Die jeweilig zutreffenden Symbole finden Sie auf dem Etikett Ihres Produktes. / **en** Please note that not all symbols shown apply to your product. The relevant symbols can be found on the product label. / **fr** Veuillez noter que tous les symboles décrits ne s'appliquent pas à votre produit. Vous trouverez les symboles correspondants sur l'étiquette de votre produit. / **it** Si prega di notare che non tutti i simboli mostrati sono applicabili al vostro prodotto. Troverete i simboli pertinenti sull'etichetta del vostro prodotto. / **sv** Observera att inte alla avbildade symboler är tillämpliga för din produkt. Vilka symboler som gäller för din produkt hittar du på produktetiketten. / **da** Bemærk, at ikke alle viste symboler er relevante for jeres produkt. De relevante symboler findes på produktmærkningen.

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TABLE OF CONTENTS

- 2 Explanation of Symbols
- 10 Product overview
- 25 The drainova® rinsing tube
- 25 Indication for use
- 25 Warnings
 - General
 - In combination with the catheter
- 27 Preparation
 - Connecting the rinsing tube and rinsing the catheter (for flushing out occlusions*)
 - Performing drainage after successful flushing
- 29 Connecting the rinsing tube to an in-house suction system
- 32 Glossary

EN

Read the complete instructions for use carefully before using the drainova® rinsing tube to flush out the catheter, to take an effusion sample for laboratory analysis, or to perform drainage.

All definitions of terms (*) can be found in the glossary page 32.

YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

DE: +49 7471 73972-0
info@ewimed.com

AT: +43 2231 22500
info@ewimed.com

EN: +49 7471 73972-0
info@ewimed.com

SE: +46 8 25 11 69
info.se@ewimed.com

CH: +41 52 577 02 55
info.ch@ewimed.com

DA: +45 31 43 50 60
info.dk@ewimed.com

**For further detailed information please visit our
website www.ewimed.com**

THE DRAINOVA® RINSING TUBE

The drainova® rinsing tube is intended exclusively for use with the drainova® catheter, the drainova® ArgentiC catheter and the PleurX™ or PeritX™ catheter. Hereinafter the above are referred to as “catheters”. The drainova® rinsing tube is hereinafter referred to as the “rinsing tube”.
Minimum purchase quantity: 1 purchase unit = 2 tubes, sterile

EN

INDICATION FOR USE

The rinsing tube is exclusively indicated for use with the aforementioned catheters for rinsing out occlusions*, for removing effusion for in vitro diagnostics, and when short-term drainage with an in-house suction system is indicated.

The catheter is indicated for implantation in adults and children with malignant or non-malignant effusions, which also allows the rinsing tube to be used in these patient groups.

WARNINGS

General

The product may only be used by medical professionals!

Do not use the product if:

- the sterile packaging is damaged
- the product is damaged
- the expiry date has passed

The disposables are intended for single use only!

The rinsing tubes may not be resterilised. The rinsing tube is delivered in a sterile condition, which is assured by suitable quality controls.

The product is intended for single use only. Reuse may lead to cross-contamination.

j.söllner AG assumes no liability for products that have been reused, resterilised or removed from damaged packaging.

Cleaning / disinfection

If effusion leaks and comes into contact with the skin, clean the skin with soap and water. For surfaces, use a suitable cleaning agent or surface disinfectant. Do not use octenisept® or disinfectants containing iodine to clean / disinfect the catheter or safety valve.

WARNINGS

In combination with the catheter

Never cut through the catheter or cut off the safety valve.

Avoid direct contact between the catheter and sharp objects.

Keep the safety valve on the catheter clean and the access tip on the rinsing tube sterile.

Make sure that these do not come into contact with non-sterile objects in order to avoid dirt or contamination. The safety valve must be disinfected before each connection to the rinsing tube.

Make sure the safety valve and access tip are fully connected before use. Take care not to tug or pull on the rinsing tube.

When connecting the access tip to the safety valve, use a sterile procedure and ensure that the connection is firm. Accidental

disconnection can result in contamination. In such cases, the safety valve must be disinfected with a fresh alcohol wipe and the disconnected material must be disposed of. A new, sterile rinsing tube must be used to continue use.

Note: Follow local, state, and federal regulations when disposing of used material. If necessary, provide a residual waste container.

PREPARATION

Connecting the rinsing tube and rinsing the catheter (for flushing out occlusions*)

The following materials are also required for a rinsing process:

- Physiological saline solution (0.9% NaCl)
 - Syringe
 - 3-way valve
 - Cannula
 - Alcohol wipes
1. Open a new rinsing tube pack. The rinsing tube in the packaging is sterile. It is important that the access tip on the drainova® rinsing tube remains sterile.
 2. Connect a 3-way valve to the rinsing tube's Luer lock port.
 3. Fill a 10 ml or 20 ml syringe (cannula) with 10 – 20 ml of a physiological saline solution.
 4. Connect the syringe to the 3-way valve.
 5. Adjust the 3-way valve so that the connection between the syringe and the rinsing tube is open.
 6. Vent the rinsing tube so it is bubble-free. To do this, hold the end of the rinsing tube in the air and carefully inject the saline solution into the rinsing tube until liquid emerges at the end.

28 | Preparation

EN

7. Hold the access tip by the holding area and, if necessary, carefully pull the protective cover off the access tip with a slight twisting movement. Dispose of the protective cover.

8. Push the access tip into the safety valve. Make sure that the safety valve and the access tip are firmly connected during the rinsing process. You can hear and feel them snap into place.

9. Flush the catheter by pressing the plunger of the syringe.
10. If necessary, saline solution can be drawn up again with the syringe.
 - 10.1 Close the rinsing tube's pinch clamp by pressing it together.
 - 10.2 Close the 3-way valve.
 - 10.3 Disconnect the syringe from the 3-way valve, refill with saline, and reconnect to the 3-way valve.
 - 10.4 Turn the 3-way valve to the open position and gently inject saline until liquid comes out of the 3-way valve.
 - 10.5 Set the 3-way valve again as described under point 5 and continue the process at point 9.
11. To end the rinsing process after successfully eliminating the occlusion*, close the rinsing tube's pinch clamp by squeezing it.

Note: After completing the rinsing process, it should be possible to easily apply saline solution and to aspirate the effusion.


Performing drainage after successful flushing

1. Make sure the rinsing tube's pinch clamp is closed, then remove the 3-way valve and rinsing tube.
2. Depending on the medical indication, carry out drainage using a drainova®, ewimed or PleurX™ drainage set. Please refer to the

instructions for use of the corresponding drainage set. If you want to drain with an in-house suction system, continue with point 2 of the section „Connecting the rinsing tube to an in-house suction system“.

Note: If saline solution can be applied easily and the effusion can be aspirated, the rinsing process was successful.

Note: If necessary, the plug-in connection can be secured with a separately available drainova® clickFix (art. no. P1100) for additional protection against accidental disconnection.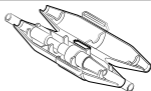

CONNECTING THE RINSING TUBE TO AN IN-HOUSE SUCTION SYSTEM


The following materials are required for connection to an in-house suction system:

- drainova® rinsing tube
- drainova® silicone cap or PleurX™/PeritX™ Valve Cap
- Hospital suction system
- Optional drainova® clickFix

1. Loosen the drainova® silicone cap or the PleurX™/PeritX™ Valve Cap:


• **With the drainova® silicone cap:**

Hold the catheter's safety valve and then carefully pull off the drainova® silicone cap. Dispose of the cap.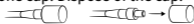

• **With the PleurX™/PeritX™ Valve Cap:**


Hold the catheter's safety valve and then twist the PleurX™/PeritX™ Valve Cap anticlockwise and gently pull it off. Discard the cap.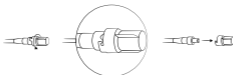


30 | Connecting the rinsing tube to an in-house suction system

2. Clean the safety valve with an alcohol-based disinfectant.

Caution: Do not use octenisept® or disinfectants containing iodine.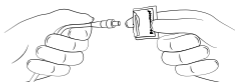

3. Use a new sterile rinsing tube. Make sure that the Luer lock port and access tip remain sterile.
4. Close the rinsing tube's pinch clamp completely by squeezing it.
5. Connect the tube attachment of the drainova® adapter (art. no. P8448) to the stepped connector of the "Thopaz+ Thoracic Drainage and Monitoring System from Medela".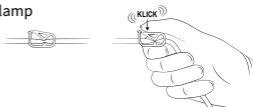

Note: Set the suction of the suction device to the medically indicated suction, up to a maximum of -20 cm H₂O or -14.7 mm Hg. If a comparable suction system is used, it is imperative that you observe the corresponding instructions for use.


6. Hold the access tip by the base and pull the protective cover off the access tip with a slight twisting motion.

7. Push the access tip into the safety valve. Make sure that the safety valve and the access tip are firmly connected. You can hear and feel them snap into place.

8. Open the rinsing tube's pinch clamp to allow drainage to begin.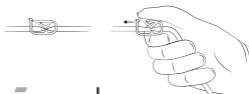


Note: The volume of the effusion and the connections with the in-house suction system should be checked at least twice a day.

Note: The indication for drainage with the hospital suction system must be adjusted in the following situations:

- the effusion is almost completely drained
- the safety valve is blocked by a clot* or a protein thread
- in event of septate effusion
- in event of empyema* or peritonitis*

Note: Observe the hygiene regulations of your facility when using the rinsing tube, especially when changing the drainage system. We recommend changing the rinsing tube every 24 hours at the latest.

9. To end the drainage, turn off the mobile suction system and close the rinsing tube's pinch clamp. Then get a new sterile drainova® silicone cap or PleurX™/PeritX™ Valve Cap ready.

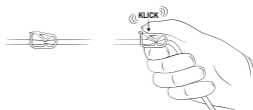10. Take the rinsing tube in one hand and the safety valve in the other hand. With a firm, steady motion, pull the access tip out of the safety valve.

11. Clean the safety valve with a suitable alcohol-based disinfectant.
12. Attach the drainova® silicone cap or the PleurX™/PeritX™ Valve Cap:


- **With the drainova® silicone cap:**

Push the new sterile drainova® silicone cap firmly onto the catheter's safety valve. The cap stays in position by itself without additional locking.

- **With the PleurX™/PeritX™ Valve Cap:**

Place the new sterile PleurX™/PeritX™ Valve Cap over the safety valve and turn it clockwise until it clicks into place.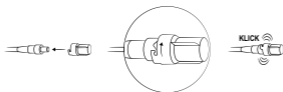

GLOSSARY

Clot	Blood clot
DEHP	Plasticizers in plastics
Empyema	Infection of the pleura
Occlusion	Closure of a hollow organ (e.g. a vessel or a medical device such as the catheter or the catheter safety valve.)
Peritonitis	Infection of the peritoneum
Phthalate	Plasticizer (e.g. DEHP)

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PleurX™ / PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
octenisept® is a trademark of Schülke & Mayr GmbH.
drainova® is a trademark and/or registered trademark of ewimed.
j. söllner AG
Cosanum AG
ewimed Switzerland AG