ewimed

DE drainova® Reservoir, Soft-Vakuum

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EN drainova® reservoir, soft-vacuum

Instructions for use

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IT drainova® serbatoio, vacuum delicato

istruzioni d'uso

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CONTENT

EN

2	Explanation of symbols	23	Connecting the drainova® Reservoir
8	Product description	24	Final steps
21	The drainova® Reservoir soft-vacuum	25	Emptying the drainova® Reservoir
21	Warnings	25	Returns/refunds
	general	26	Glossary
	in relation to your catheter		

You should have already been instructed in the correct use of the components in this set either by ewimed personnel training staff, district nursing staff or a doctor. Please contact your doctor or district nurse if you are experiencing difficulties or have questions relating to effusion drainage.

Please carefully read the instructions before using the drainova® Reservoir soft-vacuum.

All explanations of terms (*) can be found in the glossary on page 26.

YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

DE +49 7471 73972-0
info@ewimed.com

SE +46 8 25 11 69
info.se@ewimed.com

BE +49 7471 73972-0
info@ewimed.com

EN +49 7471 73972-0
info@ewimed.com

DA +45 31 43 50 60
info.dk@ewimed.com

NL +31 203 695 620
info@ewimed.com

CH +41 31 958 66 66
info.ch@ewimed.com

NO +46 8 25 11 69
info.no@ewimed.com

LU +49 7471 73972-0
info@ewimed.com

AT +43 2231 22500
info@ewimed.com

HU +36 1 998 8295
info@ewimed.hu

For further detailed information please visit our website www.ewimed.com

THE DRAINOVA® RESERVOIR SOFT-VACUUM

The drainova® reservoir soft-vacuum contains sterile material required for the safe and easy drainage of effusion fluids with your catheter.

The reservoir is CE labelled and have been sterilized with ethylene oxide (see individual packaging).

The drainova® Reservoir soft-vacuum will hereinafter be referred to as „reservoir“.

EN

WARNINGS

general

Do not use this product if:

- the sterile packaging is damaged
- damaged
- the expiry date has elapsed

The reservoir is only suitable for use for the intended purpose.

The drainage consumables are intended for single use only!

The reservoirs are delivered in sterile packaging which have undergone quality controls. The reservoir can not be re-sterilised. The product is intended for single use only. Re-use could contribute to a cross-contamination. ewimed GmbH accepts no liability for re-used, re-sterilised products or products taken from damaged packaging.

Drainage of the chest cavity

Only drain the amount of fluid prescribed by your doctor. ewimed does not recommend draining more than 1000 ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in

order to pause the flow. Possible complications relating to repeated drainage of pleural effusions* include pneumothorax*, re-expansion pulmonary oedema*, hypotension*, circulatory collapse*, septa* in the chest cavity or infection*.

Drainage of the peritoneal cavity

Only drain the amount of fluid prescribed by your doctor. ewimed does not recommend draining more than 2000 ml of fluid daily. Drainage of larger amounts should only be carried out on recommendation by your physician. You may experience an uncomfortable pulling sensation whilst draining. Should this become too intense, close the pinch clamp of the connection tube in order to pause the flow.


Possible complications relating to repeated drainage of ascites* include hypotension*, circulatory collapse*, electrolyte imbalance*, loss of protein*, hepatorenal syndrome*, leakage of ascites from incision sites, peritonitis*, septa* in the peritoneal cavity* and infection.

22 | Warnings

EN

Position of the reservoir during drainage

The drainage of fluid is assisted by the force of gravity. Therefore please ensure that the reservoir is always positioned under the catheter exit site whilst draining. During drainage, make sure that no pressure is applied to the reservoir. This could damage the one-way valve and cause contamination from the backflowing fluid.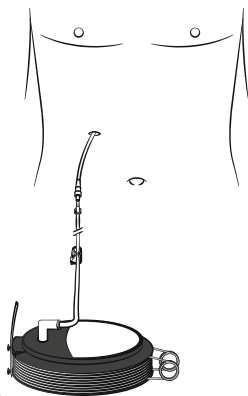

Cleaning / disinfection

Should fluid leak out and come into contact with skin, clean the skin with soap and water. Use suitable cleaning or disinfectant products for surfaces. Do not use octenisept® or products containing iodine to clean/disinfect the catheter or safety valve.

Please observe the reservoir change intervals according to the hygiene regulations of your institution.

Please note: All serious incidents relating to the product must be reported to the manufacturer and relevant authorities.

WARNINGS


in relation to your catheter

Never cut through the catheter or cut the safety valve off.

Ensure that the catheter does not come into contact with any sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i.e. is no longer attached), proceed as follows:

- firmly pinch the catheter closed with your fingers.
- using your other hand, hold the emergency slide clamp open so that the catheter can be fed through the open end of the clamp.
- push the catheter completely through the narrow end of the clamp. Please ensure that the emergency clamp is not attached directly to the valve, but approx. 2 cm away from the valve.
- Immediately inform a doctor.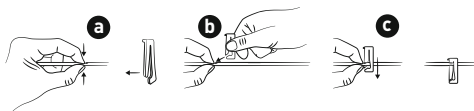

Only use original accessories.

Do not attempt to connect the catheter to non-original drainage products. Only the specially-developed access tip can be inserted into the safety valve. The use of foreign items can damage the safety valve. Incorrect handling can cause air to enter the body or effusion fluid to leak out.

Always keep the catheter safety valve clean and the access tip on the connected tube sterile.

Take care to ensure that they do not come into contact with any un-sterile objects in order to avoid contamination. The safety valve must be disinfected before each drainage.


While draining, make sure that the safety valve and the access tip are securely connected. Take care not to tug or pull on the connected tube.

Take particular care to ensure that the access tip is inserted in the safety valve in a sterile manner and make sure that the connection is secure. It can lead to contamination if the parts become disconnected by mistake. In case this occurs, disinfect the safety valve with a new alcohol wipe and dispose of the disconnected drainage material. Use a new sterile reservoir to continue with the drainage.

Please note: If required (for additional protection against accidental disconnection) the connection can be secured with the separately available drainova® clickFix (catalogue number P1100 / P1100S).

The drainova® clickFix can be additionally prescribed. It is not contained in the drainage kit.

EN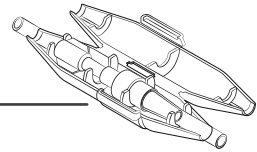

CONNECTING THE DRAINOVA® RESERVOIR

1. Remove the drainova® silicone cap, sterile or the PleurX™ / PeritX™ Valve Cap:


- **When using the drainova® silicone cap, sterile:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® silicone cap, sterile. Dispose of the cap.


- **When using the PleurX™ / PeritX™ Valve Cap:**

Firmly hold the safety valve of the catheter, turn the Valve Cap anticlockwise and carefully remove. Dispose of the cap.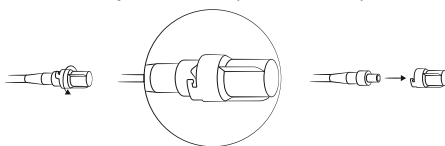

2. Hold the safety valve in one hand and clean the area around the safety valve-opening with an alcohol wipe or a suitable alcohol-based disinfectant.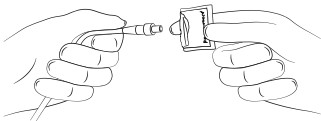


Warning: Do not use octenisept® or other disinfectants containing iodine.


3. Open the packaging of the reservoir and take it out.

The product in the packaging is sterile. It is important that the access tip on the tubing remains sterile.

Remove the tape wrapped around the coiled up tubing.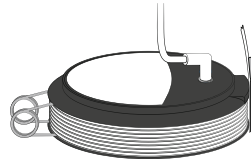

4. Hold the base of the access tip in one hand. Completely close the pinch clamp on the tube with the other hand.

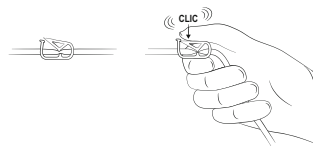
24 | Final steps

- Continue to hold the access tip at the base and carefully remove the protective film from the access tip (if necessary by gently rotating).


- 1 Insert the access tip in the safety valve.

Ensure that the safety valve and the access tip are securely connected during drainage. A click can be heard and felt when connected.

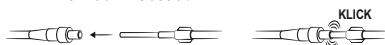
- Push down on the reservoir to remove the orange-coloured safety clasp. Keep the orange-coloured clasp for later disposal of the reservoir.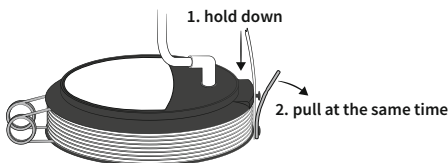

Recommendation: The duration of drainage can be shortened by placing the reservoir at a lower position.



- Open the pinch clamp on the tubing so that drainage can begin.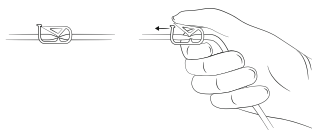

Please note: The flow can be slowed or interrupted when:

- The fluid has almost completely drained
- The fluid has a high viscosity
- The safety valve is blocked due to a clot* or a protein thread

- When you have reached the drainage amount recommended by your doctor, stop drainage by closing the pinch clamp. The top stop button shows you the drained amount.

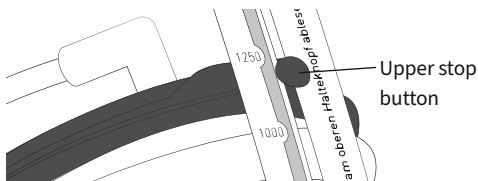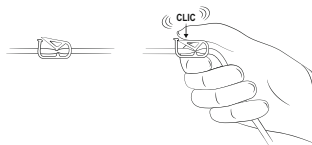FINAL STEPS

- Open the packaging of the drainova® silicon cap, sterile or the PleurX™ / PeritX™ Valve Cap. But to begin with, leave the cap in the packaging.
- Take the tubing in one hand and the safety valve in the other hand. Pull the access tip in a firm, smooth motion out of the safety valve. Place the tubing down.


- Clean the safety valve for approx. 30 seconds with an alcohol wipe or other suitable alcohol-based disinfectant and then dispose of the used wipe.

Warning: Do not use octenisept® or other disinfectants containing iodine.

- Attach either the drainova® silicon cap, sterile or PleurX™ / PeritX™ Valve Cap:


- **When using the drainova®silicone cap, sterile:**

Securely attach the new drainova® silicone cap, sterile onto the safety valve of the catheter. The cap will stay in place without the need for additional support.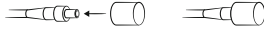

- **When using the PleurX™ / PeritX™ Valve Cap:**

Attach the new PleurX™ / PeritX™ Valve Cap onto the safety valve and turn clockwise until it clicks into place.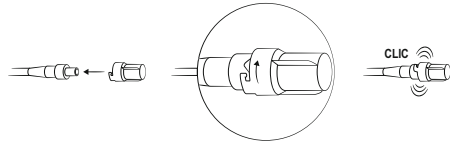

EN

EMPTYING THE DRAINOVA® RESERVOIR

1. For the following steps you will need scissors, a towel, the orange-coloured safety clasp, a plastic bag (e.g. the outer packaging in which the reservoir came in).
2. Place the reservoir on the floor. Open the pinch clamp and lift up the tube. Residual fluid in the tube can now drain off.
3. Close the pinch clamp.
4. Now cut through the tube below the one-way valve* (Fig.1). Dispose of the cut-off piece of tube with the pinch clamp and access tip. If necessary, use the laid out cloth. To dispose of the collected fluid in the toilet, position the reservoir upside down so that the tube opening is pointing downwards above the toilet. You can start the emptying process by compressing the reservoir.

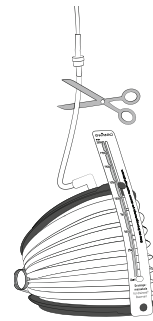
Fig. 1

Warning: Used products and effusion fluid may pose biological risks. Therefore please observe the local and national regulations when disposing of used products and fluid.

5. Once the reservoir is emptied and compressed, attach the orange safety clasp. Make sure that, where possible, there is no remaining fluid in the tube. Put the reservoir together with the drainage tubing into the plastic bag provided.
6. Dispose of the bag in accordance with the local regulations.

RETURNS/REFUNDS

Only originally packaged (unopened, non-damaged, non-labelled) boxes of reservoir will be accepted for return and refund.

GLOSSARY

EN

Ascites	Effusion or accumulation of fluid in the abdomen (stomach)
Circulatory collapse	Temporary loss of consciousness
Clot	Blood clot
DEHP	Plasticizers in plastics
Electrolyte imbalance	Disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or cells in the body
Hepatorenal syndrome	Decrease in renal function
Hypotension	Low blood pressure
Incision	Skin incision during the operation e.g. for the implantation of the catheter
Infection	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
Protein loss	Reduction of the amount of protein in the blood (protein depletion)
Peritoneal cavity	Abdominal cavity
Peritonitis	Inflammation of the peritoneum (abdominal cavity)
Phthalate	Plasticizer (e.g. DEHP)
Pleural effusion	Effusion or fluid accumulation in the pleural cavity
Pneumothorax	Air in the space between the lungs and the chest wall
Re-expansion pulmonary edema	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
One-way valve	Prevention of fluid backflow for increased patient safety
Septa	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed GmbH.
octenisept® is a trademark of Schülke & Mayr GmbH.
ewimed Netherlands B.V.
Germany | ewimed GmbH | Dorfstrasse 24 | 72379 Hechingen | Tel.: +49 7471 73972-0 | info@ewimed.com