
ewimed

drainova® clickFix

- DE** Gebrauchsanweisung
- EN** Instructions for use
- FR** Instructions d'utilisation
- IT** istruzioni d'uso
- SV** Bruksanvisning
- DA** Brugsanvisning
- NO** Bruksanvisning
- HU** Használati útmutató
- NL** Gebruiksaanwijzing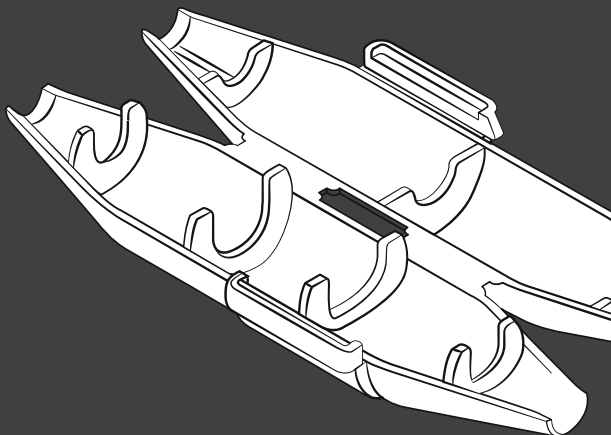

REF P1100/P1100S

DE ERKLÄRUNG DER SYMBOLE
EN EXPLANATION OF SYMBOLS
FR EXPLICATION DES SYMBOLES
IT SPIEGAZIONI DEI SIMBOLI
SV FÖRKLARING AV SYMBOLERNA

DA SYMBOLFORKLARING
NO FORKLARING AV -SYMBOLENE
HU SZIMBÓLUMOK MAGYARÁZATA
NL TOELICHTING VAN DE SYMBOLEN

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DRAINOVA® CLICKFIX

The drainova® clickFix secures the connection between a drainova®, drainova® ArgentiC, PleurX™ or PeritX™ catheter safety valve and a drainage set with compatible access tip and prevents accidental disconnection.

The drainova® clickFix is hereinafter referred to as „clickFix“. The clickFix is only suitable to be used for the intended purpose

The clickFix provides the following benefits:

- Unintentional disconnection of the drainova®, drainova®ArgentiC, PleurX™ or PeritX™ catheter safety valve and the drainage sets with compatible access tip will be prevented.
- No additional securing, e.g. with adhesive strips, necessary.
- Additional protection against contamination of the connection.

GENERAL INFORMATION

- The products (P1100 1 PU = 10 pieces, P100S 1 PU = 2 pieces) are delivered non-sterile and can be used as non-sterile products. delivered non-sterile and can be used as a non-sterile product.
- The product is a single-use product and cannot be used more than once.

AREAS OF APPLICATION

For securing the connection between a catheter safety valve and compatible access tips of (e.g.):

- 2000 drainova® reservoir, soft-vacuum
- 50-7210 PleurX™ vacuum reservoir, 1000 ml
- 50-7220 ewimed gravity reservoir, 2000 ml
- 50-7500B PleurX™ drainage kit, 500 ml
- 50-7510 PleurX™ drainage kit, 1000 ml
- 50-7505 ewimed drainage kit, 2000 ml
- 2010 drainova® reservoir, drainage set
- 50-7504 ewimed drainage kit, 400 ml
- 6060, 6060/2 drainova® flushing line
- P8531 ewimed Pneu-Pack I, 2000 ml

Please note: The connection between the drainova®, drainova® ArgentiC, PleurX™ or PeritX™ catheter safety valve and the access tip is deliberately not secured for patient safety reasons.

If the connection is secured, it cannot be disconnected in the event of unexpected force placed on the catheter. This increases the risk of damaging the catheter, pulling the catheter out of the patient or causing injury to the patient.

Some customers and patients however, want a suitable security device to prevent accidental disconnection. Previously, nursing staff has secured the connection by usually applying an adhesive strip around it. This is no longer necessary when using the clickFix.

WARNINGS

Please make sure to insert the tube connection correctly into the clickFix to avoid overly squeezing the tube.

A pinched off tube or catheter can lead to damage being caused to the drainage product or the catheter, which can then no longer be used.

If the catheter has been accidentally damaged, use the emergency clamp from the dressing material set of the PleurX™, ewimed or drainova® drainage sets to close the catheter. Please also observe the instructions and notes in the corresponding instructions for use.

The clickFix is delivered non-sterile and can be used as a non-sterile product.

As the clickFix does not come into contact with fluid-carrying surfaces, it can be used non-sterile.

Please also observe the hygiene regulations of your institution.

Please also observe the corresponding instructions for use of the drainage products.

Do not use the product if visibly damaged.

INSTRUCTION MANUAL

1. Insert the catheter safety valve with the connected drainage material into the clickFix in such a way that the connection is attached on the side with the four ribs (see fig. 1).

Please note: The direction of insertion is irrelevant.

2. Close the clickFix by simply pressing the two halves together.
3. After use, open the clasp and pull the halves apart.

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed GmbH.
ewimed GmbH
ewimed Netherlands B.V.
ewimed Switzerland AG