
ewimed

**DE drainova® Reservoir, Soft -Vakuum / EN drainova® reservoir, soft vacuum / BG Резервоар drainova®, мек вакуум / DA drainova® reservoir, blødt vakuum / EL δεξαμενή drainova®, μαλακό κενό / ES Colector drainova®, vacío suave / ET drainova® reservuaar, pehme vaakum / FI drainova®-säiliö, pehmeä tyhjiö / FR Réservoir drainova®, soft-vacuum / HR Spremnik drainova®, lagani vakuum / HU drainova® tartály, enyhe vákuummal / IT drainova® serbatoio, vacuum delicato / LT drainova® rezervuarą, Soft vakuumas / LV drainova® rezervuārs, Soft-Vakuum / MT Riżerva drainova®, vakwu artab / NL drainova® Reservoir, Soft-Vakuum / NO drainova®-beholder, soft-vakuum / PL zbiornik drainova®, miękka próżnia / PT drainova® reservatório, vácuo suave / RO Rezervor drainova®, vid ușor / SK Rezervoár drainova®, mierny podtlak / SL Rezervoar drainova®, mehki vakuum / SV drainova® reservoir, mjukt vakuum / TR drainova® rezervuar, yumuşak vakum**

**DE Gebrauchsanweisung / EN Instructions for use / BG Инструкции за употреба / DA Brugsanvisning / EL Οδηγίες χρήσης / ES Instrucciones de uso / ET Kasutusjuhend / FI Käyttöohjeet / FR Mode d'emploi / HR Upute za uporabu / HU Használati utasítás / IT Istruzioni per l'uso / LT Naudojimo instrukcijos / LV Lietošanas instrukcija / MT Struzzjoni għall-użu / NL Gebruiksaanwijzing / NO Bruksanvisning / PL Instrukcja obsługi / PT Instruções de utilização / RO Instrucțiuni de utilizare / SK Návod na použitie / SL Navodila za uporabo / SV Instruktioner för användning / TR Kullanım Kilavuzu**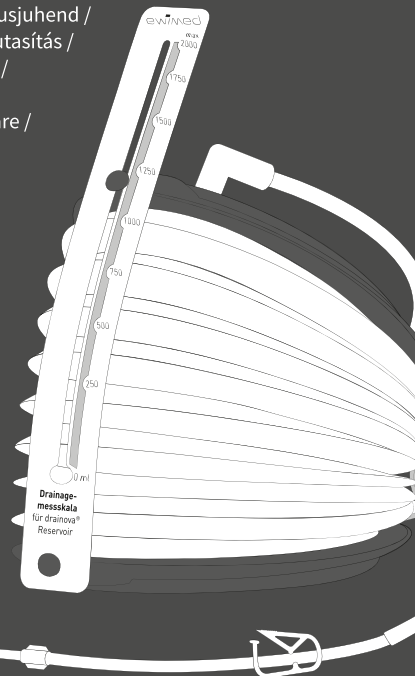

REF 2000

**2 DE KONTAKT / EN CONTACT US / BG КОНТАКТ / DA KONTAKT / EL ΣΤΟΙΧΕΙΑ ΕΠΙΚΟΙΝΩΝΙΑΣ / ES CONTACTO / ET KONTAKTANDMED / FI YHTEYS / FR CONTACT / HR KONTAKT / HU KAPCSOLAT / IT CONTATTI / LT KONTAKTINĖ INFORMACIJA / LV KONTAKTINFORMĀCIJA / MT KUNTATT / NL CONTACT / NO KONTAKT / PL KONTAKT / PT CONTACTO / RO CONTACT / SK KONTAKT / SL KONTAKTNI PODATKI / SV KONTAKTA OSS / TR İLETİŞİM**

**AUSTRIA**

ewimed austria GmbH  
Tullnerbachstrasse 92a  
3011 Neu-Purkersdorf  
Tel.: +43 2231 2250-0  
info@ewimed.com

**BELGIUM**

ewimed Belgium BV  
Kortrijksesteenweg 1126a  
9051 Sint-Denijs-Westrem  
Tel.: +32 7848 0510  
Info.be@ewimed.com

**DENMARK**

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## ENGLISH

Read the entire instructions for use carefully before using the product to drain the effusion.

All explanations of terms (\*) can be found in the glossary on page 15.

### THE DRAINOVA® RESERVOIR, SOFT VACUUM

- Intended use: The Drainova® reservoir, drainage set is used to drain fluid from the chest or abdominal cavity and for patient care.
- It serves as a collection container for drainage of recurrent or refractory effusions.
- Active suction build-up via integrated spring mechanism.

The Drainova® reservoir, soft vacuum is hereinafter referred to as "reservoir".

#### Indications:

The reservoir is indicated for use in combination with a compatible safety valve on a drainage catheter.

#### Contraindications:

The reservoir has no known contraindications.

#### Application environment:

The reservoir is intended for use in clinical facilities (such as hospitals and doctors' offices).

The reservoir is CE marked and sterilized with ethylene oxide gas (see packaging).

## WARNINGS

### General

#### Do not use the product if:

- the sterile packaging is damaged
- the product is damaged
- the expiry date has passed

The reservoir is only suitable for use for the intended purpose.

**Patient target group:** The medical device may only be used in adults.

**User group:** The medical device may only be used by adults.

#### The consumables are intended for single use only!

- The reservoir is delivered in a sterile condition, which is ensured by suitable quality controls.
- The reservoir must not be resterilized.
- The product is intended for single use only.
- Reuse may contribute to cross-contamination.
- ewimed GmbH accepts no liability for products that have been reused, resterilized, or removed from damaged packaging.

#### Drainage from the chest cavity

- Observe the drainage quantity according to the doctor's instructions.
- ewimed recommends draining no more than 1000 ml of effusion per day.
- Larger quantities should only be drained according to the doctor's instructions.
- The patient may feel an unpleasant pulling sensation during drainage. If this is the case, close the pinch clamp on the connection tube to interrupt the flow of fluid.
- Possible complications of recurrent drainage of a pleural effusion\* include pneumothorax\*, re-expansion pulmonary edema\*, hypotension\*, circulatory collapse\*, septa\* in the chest cavity, or infection\*.

#### Drainage from the abdominal cavity

- Observe the drainage quantity according to the doctor's instructions.
- ewimed recommends draining no more than 2000 ml of effusion per day.
- Larger quantities should only be drained according to the doctor's instructions.
- The patient may feel an unpleasant pulling sensation during drainage. If this is the case, close the pinch clamp on the connection tube to interrupt the flow of fluid.
- Possible complications of recurrent drainage of ascites\* include hypotension\*, circulatory collapse\*, electrolyte imbalance\*, protein loss\*, hepatorenal syndrome\*, leakage of ascites\* through the incisions\*, peritonitis\*, septa\* in the peritoneal cavity\*, and infection\*.

#### Position of the reservoir during drainage **1**

- Gravity-based drainage.
- The reservoir should always be positioned below the catheter exit site.

- Do not apply pressure to the reservoir during drainage.
- Pressure could damage the non-return valve\* and cause contamination.

#### Cleaning / disinfection

- In case of effusion leakage: Clean skin with soap and water.
- For surfaces: Use a suitable cleaning agent or disinfectant.
- Catheter / safety valve: Do not use octenisept® or disinfectant containing iodine.

Observe the reservoir change intervals in accordance with the hygiene regulations of the respective facility.

**Note for the user and/or patient:** Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## WARNINGS

### In combination with the catheter

**Never cut through the catheter or cut off the safety valve.**

Avoid contact with sharp objects.

If the function of the safety valve is no longer guaranteed or if the safety valve has been accidentally cut off or is no longer present, proceed as follows: **2**

- Squeeze the catheter firmly with your fingers.
- Open the clamp with one hand and insert the catheter into the open end.
- Use a clamp available in the clinic. Position the clamp at a distance of approx. 2 cm from the valve.
- Contact the attending physician immediately.

#### Only use compatible products

- Only compatible products may be connected to the catheter.
- Nothing other than the access tip specially developed for this purpose may be inserted into the safety valve; otherwise the safety valve may be damaged.
- Improper handling can lead to air entering the body or uncontrolled discharge of effusion.

#### Keep the safety valve on the catheter clean and the access tip of the rinsing tube on the connection tube sterile.

- Do not allow contact with non-sterile objects, in order to avoid soiling or contamination.
- The safety valve must be disinfected before each drainage.

#### Ensure that the safety valve and the access tip are fully connected during the drainage process. Ensure that the connection tube is not pulled or tugged.

- Ensure a sterile procedure when connecting the access tip to the safety valve.
- The plug connection must be firmly connected.
- Risk of contamination during disconnection. If this happens, disinfect the safety valve with a fresh alcohol swab.
- Dispose of disconnected drainage material.
- A new, sterile reservoir must be used to continue.
- Observe the local disposal regulations.

**Note:** If necessary, the plug connection can be secured with a separately available drainova® clickFix (Art. No. P1100 / P1100S) to provide additional protection against accidental disconnection.

The drainova® clickFix can be ordered separately and is not part of the reservoir. **3**

## CONNECTING THE DRAINOVA® RESERVOIR

- Loosen the sterile drainova® silicone cap or the PleurX™ / PeritX™ valve cap:
  - **with the drainova® silicone cap, sterile:**
    - Hold the safety valve of the catheter, carefully pull off the drainova® silicone cap, sterile. **4**
    - Dispose of the cap.
  - **with the PleurX™ / PeritX™ valve cap:**
    - Hold the safety valve of the catheter, turn PleurX™ / PeritX™ valve cap counterclockwise and carefully pull off. **5**
    - Dispose of the cap.
- Hold the safety valve with one hand.
  - Clean the area around the safety valve opening (e.g. with an alcohol swab or a suitable alcohol-based disinfectant). **6**
- Open the packaging and remove the reservoir. **7**
  - The reservoir in the packaging is sterile.
  - Ensure that the access tip on the connection tube remains sterile.
  - Remove the fixation which is attached to the coiled connection tube.
- Hold the access tip firmly at the base.
  - Fully close the pinch clamp on the connection tube. **8**
- Continue to hold the access tip firmly, and carefully pull the protective cover off the access tip with a slight twisting motion. **9**
  - Push the access tip into the safety valve. **10**
    - Ensure that the safety valve and the access tip are firmly connected during the drainage process.
    - You will hear and feel it click into place.
- Squeeze the reservoir to remove the orange fastener. **11**
  - Keep the orange fastener for disposal.

**Caution:** Do not use octenisept® or disinfectant containing iodine.

**Recommendation:** The drainage time can be shortened by placing the reservoir in a lower position.

- Open the pinch clamp on the connection tube to start drainage. **12**

**Note:** The flow of fluid can be slowed down or interrupted if:

  - the effusion is almost completely drained
  - the effusion has a high viscosity
  - the safety valve is blocked by a clot\* or a protein thread.
- Stop drainage when the recommended volume is reached by closing the pinch clamp. **8**
  - Read the drainage volume on the upper retaining button. **14**

## FINAL STEPS

- Open the packaging of the drainova® silicone cap, sterile or the PleurX™ / PeritX™ valve cap.
  - Leave the cap in the packaging.
- Hold the connection tube in one hand and the safety valve in the other. **15**
  - Pull the access tip out of the safety valve with a firm, smooth movement and put the connection tube down.
- Clean the safety valve with an alcohol swab or suitable alcohol-based disinfectant. **6**

**Caution:** Do not use octenisept® or disinfectant containing iodine.

- Attach the sterile drainova® silicone cap or the PleurX™ / PeritX™ valve cap:
  - **with the drainova® silicone cap, sterile:**
    - Push the new sterile drainova® silicone cap firmly onto the safety valve of the catheter. **16**
    - The cap is automatically held in place without additional locking.
  - **with the PleurX™ / PeritX™ valve cap:**
    - Slide the new valve cap onto the safety valve, turn clockwise until it locks firmly into place. **17**

## 18 EMPTYING THE DRAINOVA® RESERVOIR

1. Preparation: Prepare scissors, cloth, orange fastener, plastic bag.
2. Place the reservoir on the floor.
  - Open the pinch clamp and hold the tube up so that the residual liquid can drain out.
3. Close the pinch clamp.
4. Cut the tube below the non-return valve\* and dispose of the tube section with the pinch clamp and access tip.
  - Position the reservoir upside down over the toilet and empty it by squeezing. 18

**Note:** Used products and effusions may pose biological risks. Therefore, please observe local and national regulations when disposing of used products and effusions.

5. After emptying, fold up the reservoir.
  - Attach the orange-colored fastener. Ensure that as little liquid as possible remains in the tube.
  - Place the reservoir and connection tube in the plastic bag.
6. Dispose of in accordance with local regulations.

## RETURN OR CREDIT NOTE

Only originally packed (unopened, unlabeled, or damaged) boxes with reservoirs will be taken back and credited.

## GLOSSARY

**Ascites** – effusion or accumulation of fluid in the abdomen (stomach)  
**DEHP** – plasticizer in plastic  
**Protein loss** – reduction in the amount of protein in the blood (protein depletion)  
**Electrolyte imbalance** – disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or in the body cells  
**Hepatorenal syndrome** – decrease in kidney function  
**Hypotension** – low blood pressure  
**Infection** – reaction of the body's own defenses to pathogens such as bacteria, viruses, etc.  
**Incision** – cut in the skin during surgery, e.g. for implanting a catheter  
**Coagulum** – blood clot  
**Circulatory collapse** – temporary loss of consciousness  
**Peritoneal cavity** – part of the abdominal cavity  
**Peritonitis** – inflammation of the peritoneum (abdominal membrane)  
**Pleural effusion** – effusion or accumulation of fluid in the pleural cavity  
**Pneumothorax** – air in the space between the lungs and the chest wall  
**Re-expansion pulmonary edema** – fluid retention in the functional lung tissue due to the sudden expansion of the previously compressed lung; acute respiratory distress  
**Non-return valve** – backflow protection for increased patient safety  
**Septa** – tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)

Further, detailed information can also be found on our website at [www.ewimed.de](http://www.ewimed.de)

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PleurX™/PeritX™ is a trademark and / or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and / or registered trademark of ewimed GmbH.
octenisept® is a trademark of Schülke & Mayr GmbH.
Shaoxing Yibon Medical Co., Ltd.
OBELIS SWISS GmbH
EURASIA Deutschland GmbH