ewimed

**DE drainova® Verbandmaterial-Set / EN drainova® dressing material set / BG Комплект превързочни материали drainova® / DA drainova® Forbindingsmateriale-sæt / EL Σετ επιδεσμικού υλικού drainova® / ES Juego de apósitos drainova® / ET drainova® sidemete komplekt / FI drainova® sidetarvikeseitti / FR drainova® Kit de pansement / HR Komplet zavojnog materijala drainova® / HU drainova® kötszerkészlet / IT drainova® kit di medicazione / LT drainova® tvarstomosios medžiagos rinkinys / LV drainova® pārsienamo materiālu komplekts / MT Sett ta' materjal ta' faxxatura drainova® / NL drainova® verbandmateriaalset / NO drainova® bandasjesett / PL Zestaw materiałów opatrunkowych drainova® / PT Conjunto de material de penso drainova® / RO Set de pansamente drainova® / SK Sada obvázového materiálu drainova® / SL Komplet obvez drainova® / SV drainova® uppsättning förbandsmaterial / TR drainova® bandaj malzemesi seti**

**DE Gebrauchsanweisung / EN Instructions for use / BG Инструкции за употреба / DA Brugsanvisning / EL Οδηγίες χρήσης / ES Instrucciones de uso / ET Kasutusjuhend / FI Käyttöohjeet / FR Mode d'emploi / HR Upute za uporabu / HU Használati utasítás / IT Istruzioni per l'uso / LT Naudojimo instrukcijos / LV Lietošanas instrukcija / MT Struzzjoni għall-użu / NL Gebruiksaanwijzing / NO Bruksanvisning / PL Instrukcja obsługi / PT Instruções de utilização / RO Instrucțiuni de utilizare / SK Návod na použitie / SL Navodila za uporabo / SV Instruktioner för användning / TR Kullanım Kılavuzu**

**REF** 8020

**2 DE KONTAKT / EN CONTACT US / BG KOHTAKT / DA KONTAKT / EL ΣΤΟΙΧΕΙΑ ΕΠΙΚΟΙΝΩΝΙΑΣ / ES CONTACTO / ET KONTAKTANDMED / FI YHTEYS / FR CONTACT / HR KONTAKT / HU KAPCSOLAT / IT CONTATTI / LT KONTAKTINĖ INFORMACIJA / LV KONTAKTINFORMĀCIJA / MT KUNTATT / NL CONTACT / NO KONTAKT / PL KONTAKT / PT CONTACTO / RO CONTACT / SK KONTAKT / SL KONTAKTNI PODATKI / SV KONTAKTA OSS / TR İLETİŞİM**

**AUSTRIA**

ewimed austria GmbH  
Tullnerbachstrasse 92a  
3011 Neu-Purkersdorf  
Tel.: +43 2231 2250-0  
info@ewimed.com

**BELGIUM**

ewimed Belgium BV  
Kortrijksesteenweg 1126a  
9051 Sint-Denijs-Westrem  
Tel.: +32 7848 0510  
Info.BE@ewimed.com

**DENMARK**

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**Read the entire instructions for use carefully before using the drainova® dressing material set.**

All explanations of terms (\*) can be found in the glossary on page 15.

## THE DRAINOVA® DRESSING MATERIAL SET

- Intended use: The drainova® dressing material set contains the necessary accessories for proper dressing of catheter exit sites and for absorbing body fluids after pleural effusion or ascites drainage.
- The drainova® silicone cap, sterile or PleurX™ / PeritX™ valve cap are used to seal these catheters.
- The set is sterilized with ethylene oxide gas.
- The sterile alcohol swabs (wipes) are included separately.

The drainova® dressing material set is referred to below as the "dressing set."

### We also recommend the following for changing the dressing set:

- Alcohol-based skin disinfectant spray
- Hand disinfectant
- Non-sterile disposable gloves

### Indications:

The dressing material is intended for hygienic covering, fixation and padding of catheter exit sites and for absorbing body fluids.

### Contraindications:

The dressing set has no known contraindications.

## WARNINGS

### General

**The dressing set is only suitable for use for the intended purpose.**

**Patient target group:** The medical device may only be used on adults

**User group:** The medical device must only be used by adults.

**Application environment:** The dressing material set is intended for use in clinical facilities (such as hospitals, medical practices), nursing homes and in domestic settings.

### Do not use the product if:

- the sterile packaging is damaged
- the product is damaged
- The expiry date of the set or the included alcohol swabs (wipes) has passed

### The consumables are intended for single use only!

- The dressing set is delivered in a sterile condition, which is ensured by suitable quality controls.
- The dressing set must not be resterilized.
- Reuse may contribute to cross-contamination.
- ewimed GmbH and the compiler of the set Fuhrmann GmbH assume no liability for products that have been reused, resterilised or removed from damaged packaging.

### Cleaning / disinfection

- In case of effusion leakage: Clean skin with soap and water.
- For surfaces: Use a suitable cleaning agent or disinfectant.
- Catheter/safety valve: Do not use octenisept® or disinfectant containing iodine.

**Note:** All serious incidents associated with the use of this product must be reported to the manufacturer/compiler of the set and the competent national authorities in the member state in which the user/patient is resident.

## In combination with your catheter

**Never cut through the catheter or cut off the safety valve.**

Avoid contact with sharp objects.

If the function of the safety valve is no longer guaranteed or the safety valve has been accidentally cut off or is

no longer present, proceed as follows: **1**

- a) Press the catheter firmly closed with your fingers.
- b) Open the clamp with one hand and insert the catheter into the open end.
- c) Use a clamp available in the clinic. Position the clamp at a distance of approx. 2 cm from the valve.
- d) Contact the attending physician immediately.

## PREPARATION

1. Prepare a clean, clear work area.
2. Wash hands thoroughly with soap and water and then disinfect them.
3. If possible, put on non-sterile disposable gloves (not part of this set)
4. Remove the foil dressing over the catheter.
  - Carefully loosen one corner of the foil dressing and slowly pull it off the skin.
  - Make sure that the catheter is not accidentally pulled.

**Note:** In the event of swelling, signs of inflammation or fluid accumulation at the exit point of the catheter or at the removed skin interface, inform the attending physician.

5. If necessary, remove gloves and disinfect hands again.
6. Open the sterile packaging of the dressing set.

**Note:** Follow local, state, and federal regulations when disposing of used material. If necessary, provide a residual waste container.

## USING THE DRESSING SET

1. Place the dressing set on the work area with the flap facing upwards, **2**
  - Carefully unfold the cover cloth.
  - The items are sterile and should not come into contact with non-sterile objects.
2. Grasp the gloves at the opening and put them on. Do not touch any non-sterile objects! **3**
  - 2.1 Open the alcohol swab (wipe) packaging (included separately) on one side, and leave the swab in the packaging.
    - Place the swab on the edge of the cover cloth
3. Loosen the drainova® silicone cap, sterile or the PleurX™ / PeritX™ valve cap:
  - **with the drainova® silicone cap, sterile:**
    - Hold the safety valve of the catheter, carefully pull off the drainova® silicone cap, sterile.
    - Dispose of the cap. **4**
  - **for PleurX™ / PeritX™ valve cap:**
    - Hold the safety valve of the catheter, turn PleurX™ / PeritX™ valve cap counterclockwise and carefully pull off.
    - Dispose of the cap. **5**
4. Clean the safety valve with an alcohol swab. **6**
  - Observe exposure time of 30 seconds.

**Caution:** Do not use octenisept® or disinfectant containing iodine.

5. Open the packaging of the drainova® silicone cap, sterile and drop onto the cover cloth. **7**
  6. Attach a new drainova® silicone cap, sterile, and push it firmly onto the safety valve of the catheter. **8**
- The cap holds independently without additional locking.
  - Drainage is performed according to the separate instructions for use of the drainage set.

**Note:** Drainage is performed according to the separate instructions for use of the ewimed, drainova® or PleurX™ drainage set.

7. Clean the area around the catheter exit point with the alcohol-based skin disinfectant spray. **9**
- Allow a contact time of 30 seconds.
  - Do not apply the new dressing until completely dry.
8. Place the foam slit dressing around the catheter with the slit facing upwards.
  9. Roll up the catheter and place it on the foam slit compress.
  10. Cover the catheter with the gauze compresses.
  11. Remove gloves.
  12. The foil dressing has a carrier foil on both sides:
    - Printed backing film:
      - Consists of a middle section and two side edge strips
      - Underneath is the self-adhesive side of the dressing
    - Unprinted backing film:
      - Consists of a middle section and an outer frame
  13. Remove the middle section of the unprinted backing film.
  14. Remove the middle section of the printed backing film to expose the adhesive side.
    - To do this, bend the side edge strips slightly.
  15. Hold the foil dressing at the edge strips.
    - The window on the unprinted side is for better alignment.
    - Place the foil dressing in the middle over the catheter and stick it in place.
  16. Remove the frame of the unprinted carrier foil.
    - Start at the perforated area on the edge and peel off in a circle.
    - Smooth out the foil dressing.
  17. Use one hand to fix the foil dressing near the edge strips.
    - With the other hand, tear off the two edge strips at the same time.
    - The dressing is perforated at this point so that it tears along with the backing film.
    - Finally, smooth out the film dressing again.

**Note:** Dispose of the used dressing material in accordance with local regulations.

## FREQUENTLY ASKED QUESTIONS

### When can the stitches be removed?

The skin stitch can be removed after 7 - 10 days. However, the retaining stitch should only be removed after 30 days.

### Can I shower or bathe with the catheter?

If the catheter has been completely and correctly covered with the self-adhesive film dressing, you can take a shower. If the gauze compresses still get wet while showering, remove the dressing, clean and dry the area, and apply a new dressing. The catheter must not be immersed in water in a bath or swimming pool. Avoid direct contact of the valve with bath or shower water.

**Note:** We recommend showering before the drainage / dressing change so that the dressing can be changed promptly if it gets wet.

### How do I find out if I have an infection\*?

Inform a doctor immediately if you experience pain, reddening of the skin (erythema\*), tangible warmth, swelling (edema\*), fever, or fluid accumulation, as these may be signs of an infection\*.

Proceed in the same way if changes in the color of the effusion become visible.

Redness and slight pain are to be expected shortly after implantation, but these should not persist or worsen.

### What should I do if I accidentally pull out the catheter?

The catheter should fuse with the subcutaneous\* skin tissue through the polyester sleeve. If the catheter is accidentally pulled out and the cuff is visible, cover the catheter exit site with a sterile dressing and contact the implanting clinic immediately.

## RETURN OR CREDIT NOTE

Only originally packed (unopened, unlabeled, and undamaged) boxes of dressing sets will be accepted for return and credited.

## GLOSSARY

**Ascites** – effusion or accumulation of fluid in the abdomen (stomach)

**Erythema** – reddening of the skin

**Infection** – reaction of the body's own defenses to pathogens such as bacteria, viruses, etc.

**Edema** – Swelling due to excessive fluid accumulation

**Pleura** – Thin, serous skin in the chest cavity (pleura)

**Subcutaneous tissue** – tissue under the skin

Further, detailed information can also be found on our website at [www.ewimed.de](http://www.ewimed.de)

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PleurX™/PeritX™ is a trademark and /or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed.
octenisept® is a trademark of Schülke & Mayr GmbH.
Albo-Healthcare GmbH
ewimed Switzerland AG